Espinier Tissue Retrieval System


Instructions for Use Master E-Sac (WG)

Fannin UK Ltd,
T/A Espiner Medical
Yeo Bank 3, Kenn Rd,
Clevedon, Somerset,
BS21 6TH, UK

T: +44 (0) 1275 878 801
F: +44 (0) 1275 878 354
E: mail@espinermedical.com

CE
E334/1 0120

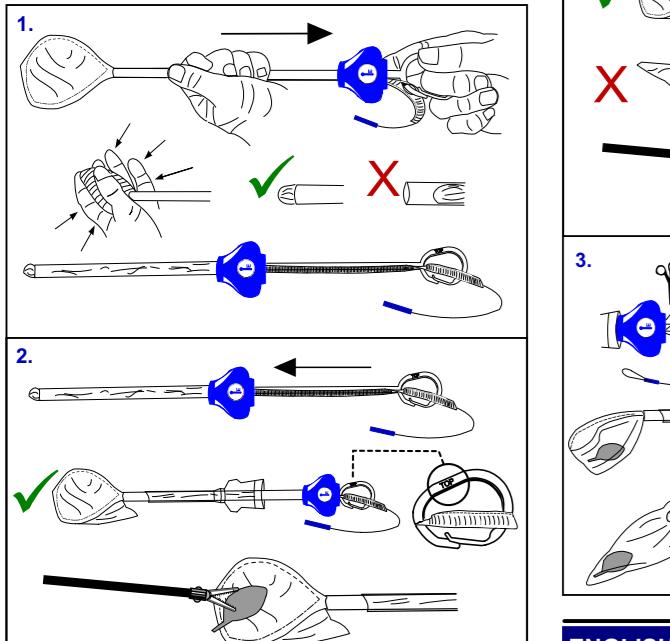Illustration A: Master E-Sac (WG)

ENGLISH (GB)

DEVICE DESCRIPTION:

Device Name: The Espiner Tissue Retrieval System

The Espiner Tissue Retrieval System is designed to facilitate the safe and secure removal of tissues or organs in the process of laparoscopic surgery. They are supplied sterile (ISO 11135) with a 5 year shelf life.


INDICATIONS FOR USE:

Retrieval System consists of a family of impervious sacs, which are sterile single use devices, that can be used alone, or with a dedicated introducer system, for the capture and removal of an organ, tissue or fluid from the body cavity during laparoscopic surgery.

CONTRAINDICATIONS:

There are no Contraindications for this device.

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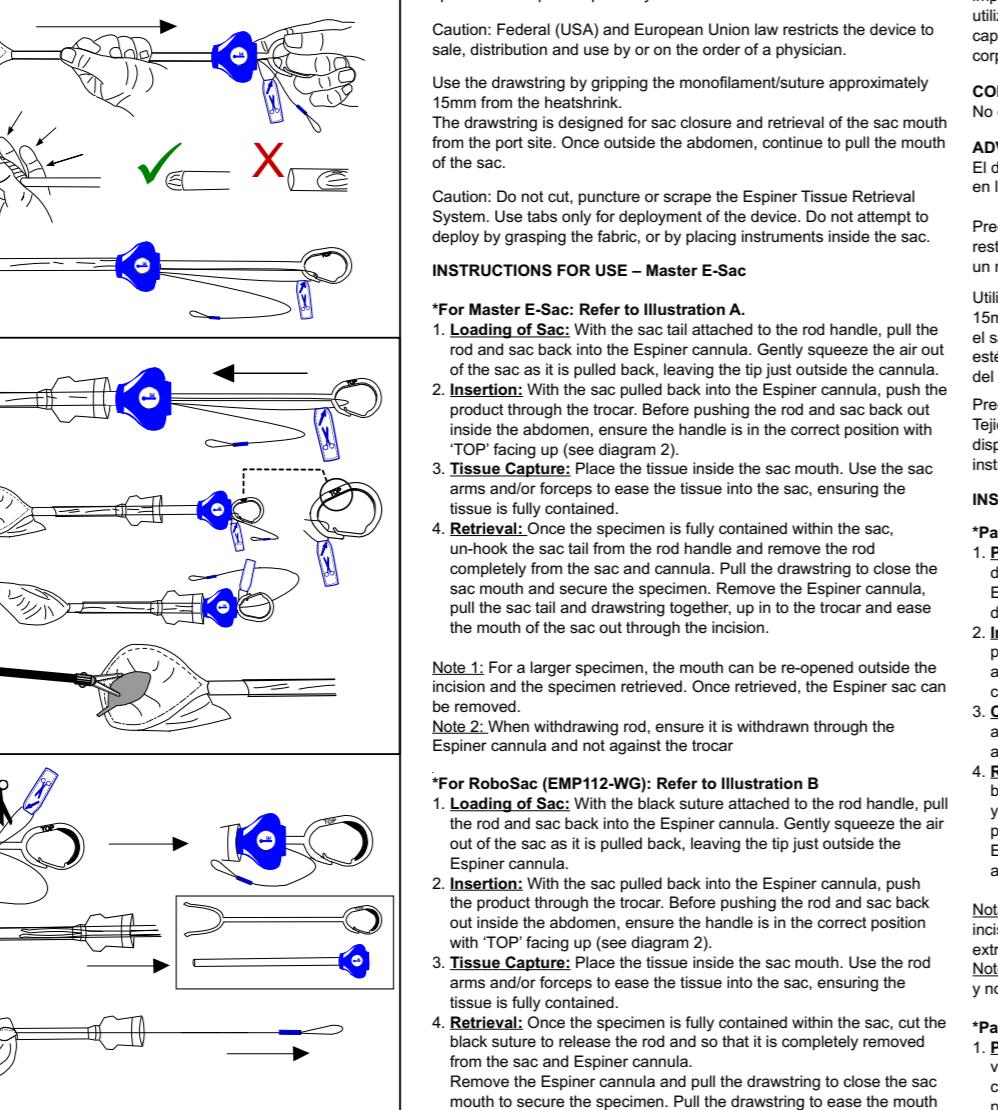CONTRAINDICACIONES:

There are no Contraindications for this device.

WARNINGS & PRECAUTIONS:
The device is only to be used by surgeon who are experienced with the specific techniques required by these medical devices.

Caution: Federal (USA) and European Union law restricts the device to capture and extraction of an organ, tissues, or fluids of the cavity corporal durante las cirugías laparoscópicas.

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Caution: Do not cut, puncture or scrape the Espiner Tissue Retrieval System. Use tabs only for deployment of the device. Do not attempt to deploy by grasping the fabric, or by placing instruments inside the sac.

INSTRUCTIONS FOR USE – Master E-Sac

*For Master E-Sac: Refer to Illustration A.

1. Loading of Sac: With the sac tail attached to the rod handle, pull the rod and sac back into the Espiner cannula. Gently squeeze the air out of the sac as it is pulled back, leaving the tip just outside the cannula.

2. Insertion: With the sac pulled back into the Espiner cannula, push the product through the trocar. Before pushing the rod and sac back out inside the abdomen, ensure the handle is in the correct position with 'TOP' facing up (see diagram 2).

3. Tissue Capture: Place the tissue inside the sac mouth. Use the sac arms and/or forces to ease the tissue into the sac, ensuring the tissue is fully contained.

4. Retrieval: Once the specimen is fully contained within the sac, un-hook the sac tail from the rod handle and remove the rod completely from the sac and cannula. Pull the drawstring to close the sac mouth and secure the specimen. Remove the Espiner cannula.

5. Note: For a larger specimen, the mouth can be re-opened outside the incision and the specimen retrieved. Once retrieved, the Espiner sac can be removed.

Note 2: When withdrawing rod, ensure it is withdrawn through the Espiner cannula and not against the trocar

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1. Loading of Sac: With the black suture attached to the rod handle, pull the rod and sac back into the Espiner cannula. Gently squeeze the air out of the sac as it is pulled back, leaving the tip just outside the cannula.

2. Insertion: With the sac pulled back into the Espiner cannula, push the product through the trocar. Before pushing the rod and sac back out inside the abdomen, ensure the handle is in the correct position with 'TOP' facing up (see diagram 2).

3. Tissue Capture: Place the tissue inside the sac mouth. Use the rod arms and/or forces to ease the tissue into the sac, ensuring the tissue is fully contained.

4. Retrieval: Once the specimen is fully contained within the sac, cut the black suture to release the rod and so that it is completely removed from the sac and Espiner cannula.

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