

EvaQMax

Smoke Free Evacuation


Procedure Operating Modes

The desired procedure can be set simply by touching the corresponding image. Automatically customizing suction default level based on desired surgical procedure.



Longer lasting filter life (up to 50 Hours)

Can save on running costs.

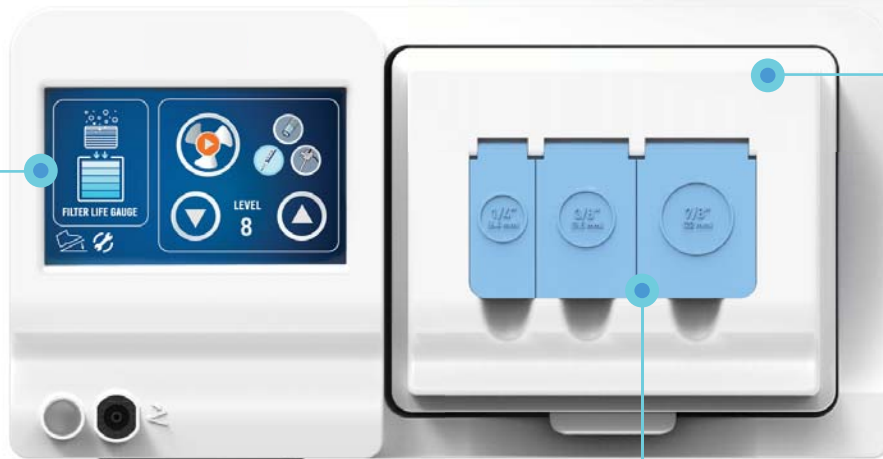


Touch screen

The touch screen panel can be operated while wearing gloves.



Multi Port design

Three different sized connection ports are designed to create an air-tight bond with the desired port. The port cover cap eliminates any possible contamination with the glove when connecting the hose to the unit.

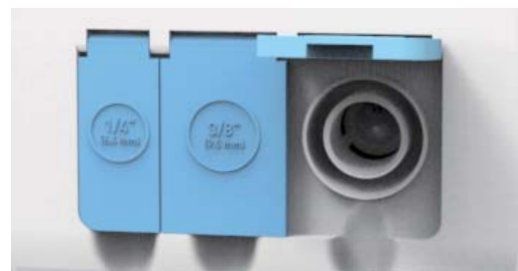

Technical Specification

| Item | Specification |
|-------------------------------|--|
| Noise Levels (Nominal Values) | maximum power < 55 dBA |
| Dimensions (H x W x D) | 6 in x 11 in x 15.5 in (15.2 cm x 27.9 cm x 39.4 cm) |
| Flow Rates | 36 cfm (708 lpm) using 7/8 in (22 mm) tubing |
| Weight | With filter installed: 12.0 lbs (5.0 kg) Without filter installed: 9.5 lbs (4.0 kg) |
| Certifications | CE, FDA |
| Warranty | Three (3) Year Full Warranty |

Accessories



Footswitch (Included)



Automatic Activation Device (Optional)



Fluid Trap (Optional)




| Model | Product Description | Packaging Info. |
|---------|-----------------------------------|-----------------|
| EQM-A | Automatic Activation Device (AAD) | 1 pcs/Box |
| EQM- FT | Fluid Trap | 20 pcs/Box |
| EQM-CAF | Filter for EQM-U | 4 pcs/Box |

Telescopic Smoke Evacuation Pencil

ONE TOUCH BUTTON EXTENDABLE TELESCOPIC TYPE

The easiest way to extend the length

1. Ability to keep focus on the surgery area even when extending the length.
2. Easy extension and retraction of suction tube with blade attachment.
3. Thin, light weight design provides comfortable usage for surgeons.
4. Superior suction Capability.

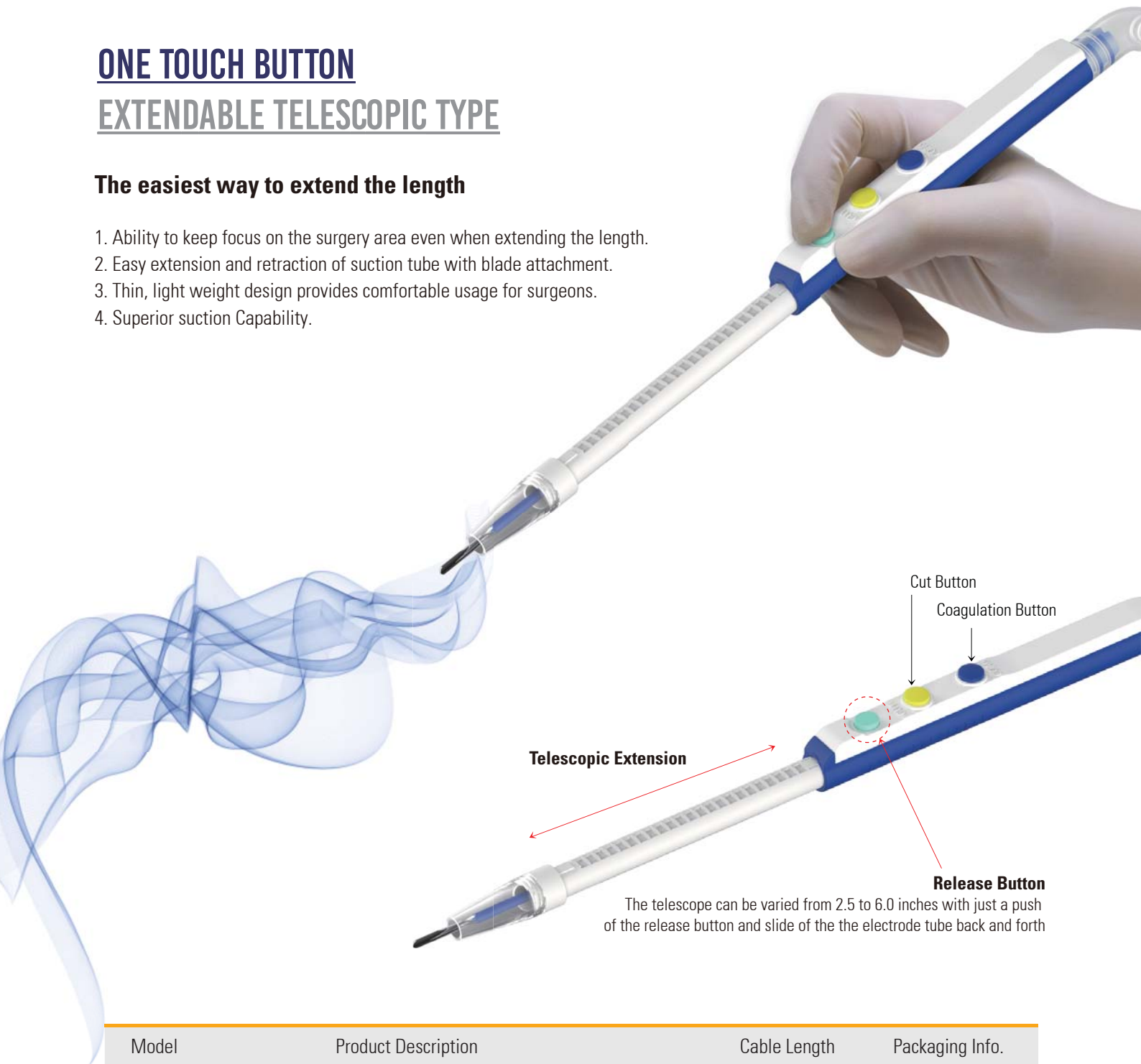


| Model | Product Description | Cable Length | Packaging Info. |
|-------|--|--------------|---|
| EvBNH | Telescopic, 2.5" Non-Stick Ceramic Coated Blade, Holster | 10' Cable | 1pc / pack 10 pcs/ Box 4 Boxes / Case |

Dual Extension Smoke Evacuation Pencil



ULTIMATE SOLUTION for SMOKE FREE in OR DUAL EXTENSION SMOKE EVACUATION PENCIL

1. EASY TO EXTEND THE TELESCOPIC TUBE WITH THE RELEASE BUTTON

The telescopic extendable tube is convenient and easy to adjust to desired length.

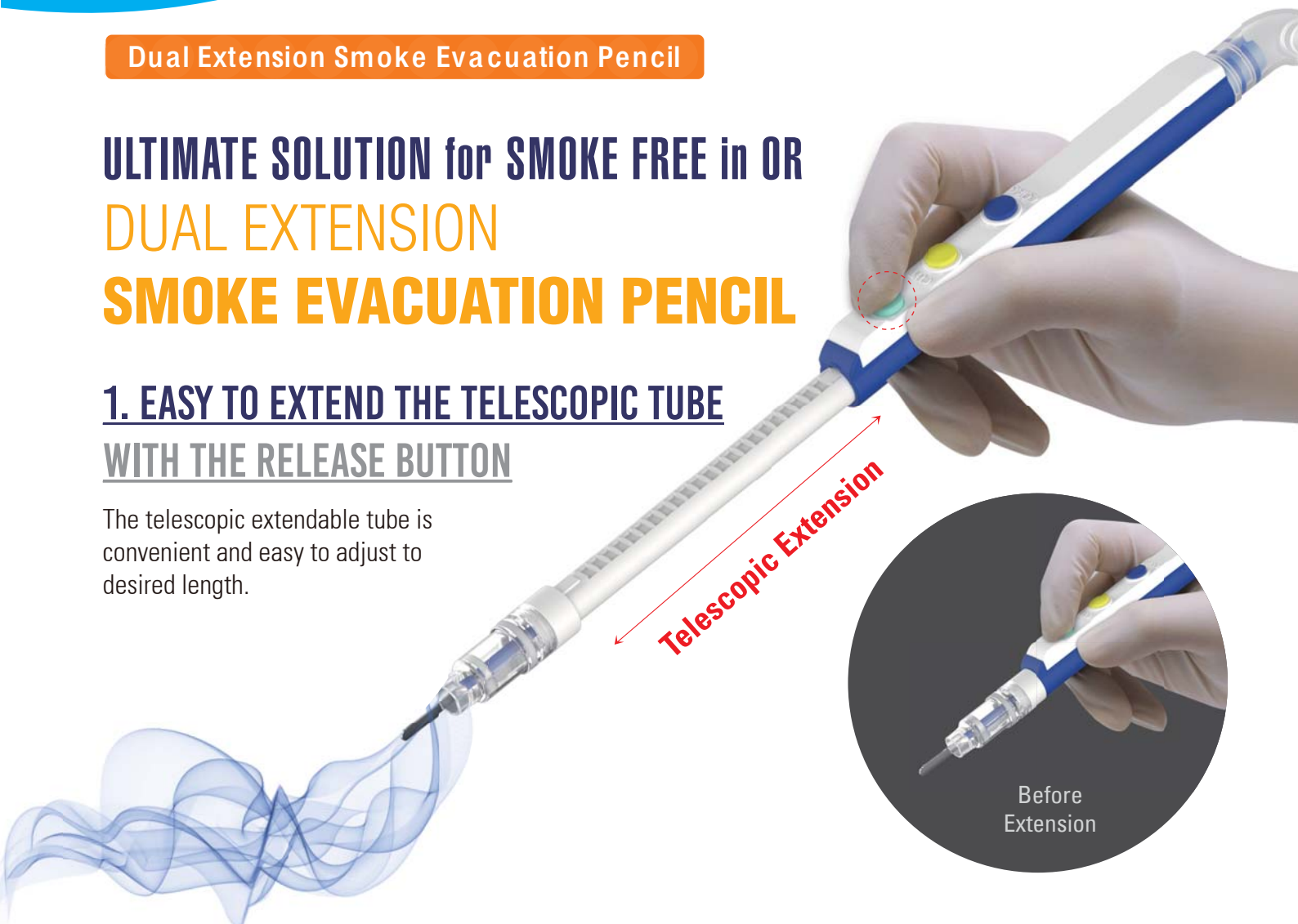


2. EXTENSION OF THE SUCTION TIP ALLOWS MAXIMUM SMOKE EVACUATION

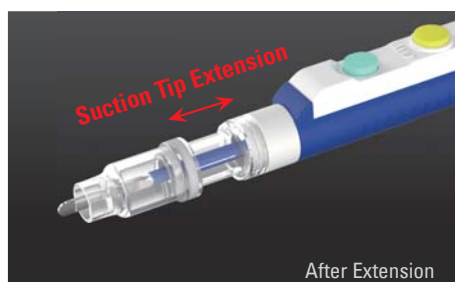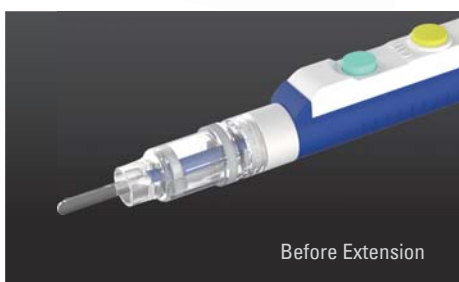





Hold the suction tip and slide it as close as possible to the blade in order to maximize the suction capacity.

| Model | Product Description | Cable Length | Packaging Info. |
|-------|--|--------------|---|
| EvDE | Dual Extension, Telescopic, 2.5" Non-Stick Ceramic Coated Blade, Holster | 10' Cable | 1pc / pack 10 pcs/ Box 4 Boxes / Case |

Surgical Smoke Evacuation Adapter

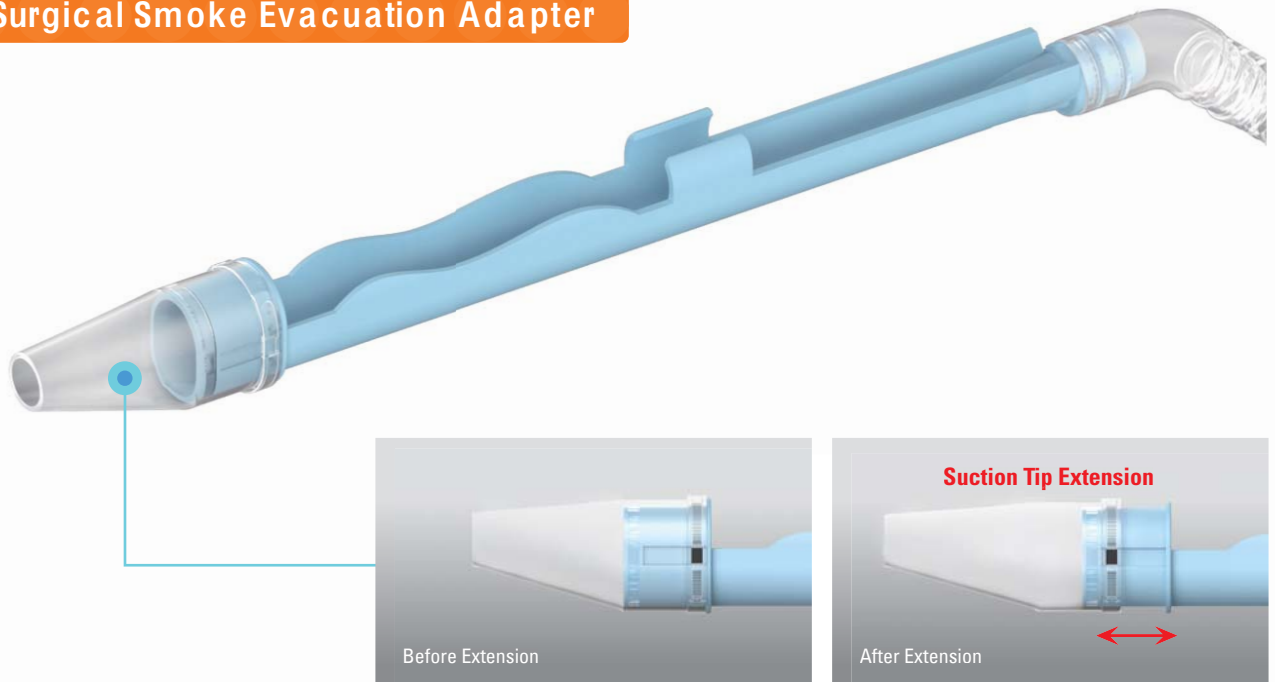

EXTENSION OF THE SUCTION TIP ALLOWS MAXIMUM SMOKE EVACUATION

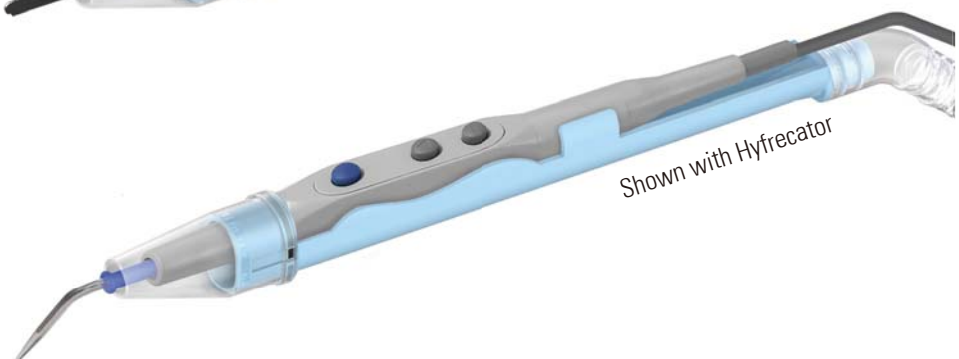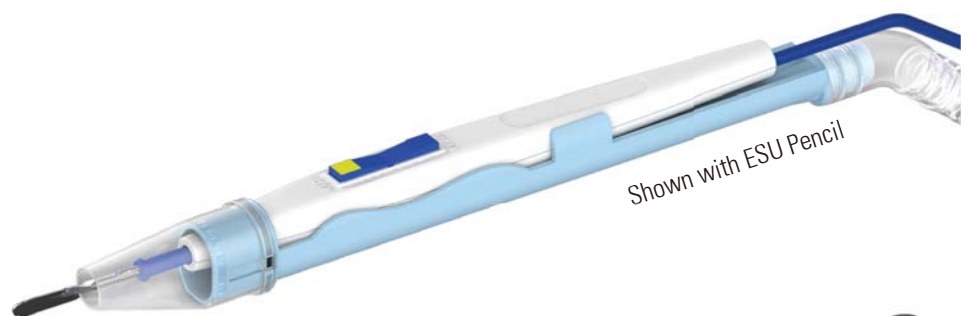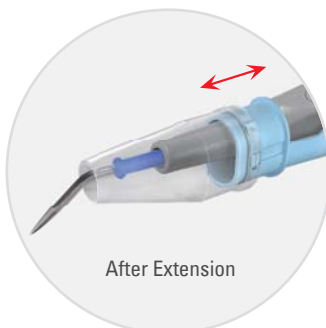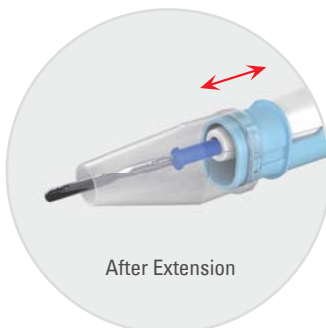


| Model | Product Description | Tubing Length |
|-------|--|-----------------------|
| EvFIT | Ideal for surgical smoke evacuator applications, Extension | 3/8in X 10' Ft Tubing |

Standard Smoke Evacuation Pencil - PS07

Compatible with any electrode on the market, including hex-shaped electrodes. Compact design and low-profile front end enhances visibility of the surgical field. Transparent nozzle minimizes any obstruction of the blade's tip or target tissue. Nozzle X-Ray identifiable

Cord set housed within the tubing to eliminate clutter at the surgical site

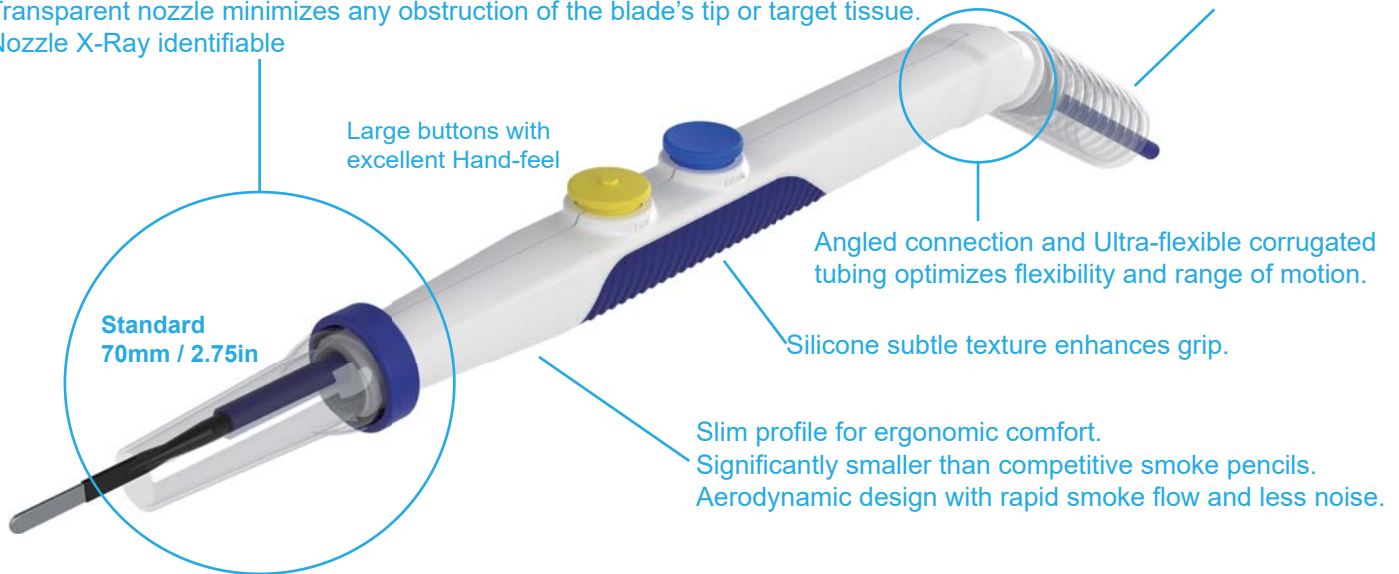

| Model | Product Description | Box Quantity |
|------------|---|--------------|
| PS07-K70GY | 70mm Non-stick Blade electrode,S nozzle 3.0m(10 ft) Tubing, 22mm connector, 3.0m(10 ft) Cord | 20PCS/Box |

Telescopic Smoke Evacuation Pencil - PS11

Position the smoke nozzle anywhere without changing of the electrode

No additional lengths of electrodes or different size smoke evacuation nozzles are needed.

Square locking mechanism prevents electrode from moving during operation

Electrode is significantly more stable at maximal length position than competitive products

Compact design and low-profile front end enhances visibility of the surgical field.

Transparent nozzle minimizes any obstruction of the blade's tip or target tissue.

Angled connection and Ultra-flexible corrugated tubing optimizes flexibility and range of motion.

Large buttons with excellent Hand-feel

Cord set housed within the tubing to eliminate clutter at the surgical site

Subtle texture enhances grip.

100mm

22mm smoke evacuator connector

3-pin plug


| Model | Product Description | Box Quantity |
|------------|---|--------------|
| PS11-K50GY | 50mm Non-stick blade electrode, Telescopic Function with locking mechanism 3.0m(10 ft) Tubing, 22mm connector, 3.0m(10 ft) Cord | 20PCS/Box |

Smoke Evacuation Accessories

Smoke Evacuation Liquid Trap

Catalogue # P-FT1

- 1) 22mm male inlet for Smoke Evacuation Pencil
- 2) Liquid Trap
- 3) Sponge liquid collector (50ml)
- 4) 22mm female outlet for Smoke Evacuator

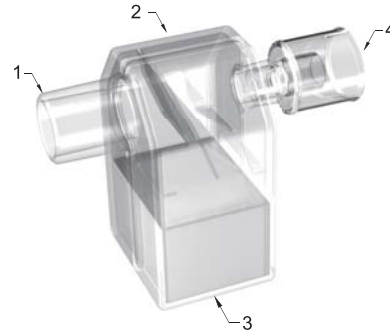

Surgical Smoke Evacuation Tubing

Catalogue # ESE-TS4

Laparoscopic, 3 meters,
 With regulator
 Sterile
 20PCS/Box



Male Luer connection



Suction regulator from 0%-100%
 connection



7/8 in (22 mm) female
 connection

Passive Smoke Evacuation Filter

Catalogue # PSF0004

NHS SC PVC Code FGB087 25PCS/Box



- Cost effective solution
- Sterile - Single use - Easy peel pouch
- Easy to use: Attached to the trocar by universal leak free luer connector
- Activated Charcoal / Bacterial/Viral Filter - 99.999% efficiency
- Thumb Wheel control
- Latex free
- Recommended for use during minimally invasive surgery involving insufflation, electrocautery, ultrasonic scalpel & laser

Fannin UK Ltd.

Fernhouse - Suite 3 Daniels Industrial Estate
 Bath Road, Gloucestershire
 GL5 3TJ, United Kingdom
 T: +44 (0) 1453 874 130
 E: salesll@fannin.eu
 W: www.fannin.eu