

## Appeel® spray makes adhesive removal quick and painless.

Just spray around the edge of the adhesive and wait a few seconds:

- No sting formula
- Can be sprayed at any angle, allowing easy access to hard to reach areas
- Contains a moisturiser
- No need to wash off
- Dries quickly, leaving surface ready for next adhesion
- Silicone based



**Silicone is kind to the skin and not absorbed by the body. After being applied, it evaporates in minutes without leaving a residue.**

### Before use

- As with all new products, **Appeel®** should be tested on a small area of skin before use on damaged skin. (To date, no adverse reactions with **Appeel®** have been reported)
- Adhesive products should be used carefully on fragile skin. Always test the removal of products with **Appeel®** on a small area of skin prior to covering large areas of very sensitive skin with adhesive products.



FOLD

**Appeel®**  
NO STING

The most advanced  
medical adhesive remover

- Easy-to-use aerosol – sprays at any angle
- Contains a moisturiser to care for the skin
- No need to wash off
- Once dry, won't affect the adhesion of dressings, sheaths and pouches



**Appeel®** is no sting – even on sore skin!

#### Appeel® no sting medical adhesive remover

Presentation	Order code	PIP code
Appeel® 50 ml aerosol	3500	311-6589
Appeel® wipes (30 wipes)	3505	311-6571

To order **Appeel®** please call CliniMed® on

**0800 585 241**

Appeel® and CliniMed® are registered trademarks of CliniMed (Holdings) Ltd. CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY Tel: 01628 850100

Email: [enquiries@clinimed.co.uk](mailto:enquiries@clinimed.co.uk)  
www.clinimed.co.uk  
902/0607

**CliniMed®**

**Appeel®**  
NO STING

FOLD

FOLD

this  
won't  
might  
hurt!

FOLD

# Appeel®

NO STING

## The most advanced medical adhesive remover available on NHS Prescription

Removing dressings, sheaths or stoma pouches can be both painful and damaging to skin. When fragile skin is damaged or peri-wound skin is stripped, it can lead to extended healing time.

Alcohol adhesive removers are traditionally used to remove adhesives, but alcohol is known to sting

**Appeel®** – a silicone based no sting medical adhesive remover

- Removes any adhesive from any part of the body easily and painlessly
- Is gentle enough to use on sensitive skin such as babies or conditions like Epidermolysis Bullosa (a blistering skin condition)



## Appeel® is no sting – even on sore skin!

**Appeel® can help remove any dressing and many products stuck to the skin:**

- Adhesive dressings
- Sheaths
- Sticky plasters
- Ostomy pouches
- ECG electrodes
- Nicotin/HRT patches

**Appeel® is suitable for use on sensitive or fragile skin.**

## How to use Appeel®

### Absorbent dressings

Spray **Appeel®** on top and around the edge of the dressing and wait a few seconds. Gently peel back the dressing, applying additional **Appeel®** if needed.



### Waterproof dressings & electrode pads

**Appeel®** will not soak through waterproof dressings; therefore, gently peel back the corner of the dressing, spray **Appeel®** around the edge and wait 30 seconds.



Continue to peel back the dressing, applying more **Appeel®** as necessary.

Before applying another adhesive product, wait 30 seconds for **Appeel®** to evaporate.



## Using Appeel® on the face

To avoid contact with the eyes, use **Appeel®** wipes or spray a small amount of **Appeel®** into the aerosol lid and drip onto the adhesive. This will have to be done quickly, before the **Appeel®** evaporates.



### Incubator use in neonates

Spray **Appeel®** into the aerosol lid, drip onto a cloth and wipe onto the adhesive dressing.

It is important to remove any residue to create a good seal between the adhesive and the skin.

**Appeel® moist wipes make removal of adhesive residue easy:**

- No sting formula
- Removes adhesive residue in seconds
- No need to wash skin after use
- Contains a moisturiser to care for the skin
- Leaves skin clean, fresh and ready for the next device or dressing application



Using **Appeel®** you can gently wipe away adhesive residue without damaging skin.

# Appeel®

NO STING