

# LBF<sup>®</sup> Hydrosolve Flushable Barrier Film Wipes

---

## Product Description

LBF Hydrosolve Flushable Barrier Film Wipes are flushable wipes containing a silicone polymer dissolved in a non-stinging solvent, which provides a protective barrier from the harmful effects of urine and faeces. They are fast-drying, which allows adhesive appliances to be re-applied shortly after use. The wipe is made using a dispersible fabric which disintegrates in seconds when placed in water. The wipe can be disposed of in the toilet bowl and flushed away.

## Indications

LBF Hydrosolve Barrier Film Wipes are indicated for the protection of intact skin that is at risk or likely to be compromised by corrosive bodily fluids (including urine, faeces, digestive enzymes), wound drainage, adhesives and friction. The product may be used on adults, children and infants. For external use only.

## Contraindications

Do not use on mucosal membranes and broken skin. It is not to be used on infected skin or in cases where there is a known sensitivity to any of the ingredients.

## Caution

LBF Hydrosolve Flushable Barrier Film in liquid form is flammable; use in a well ventilated area. Avoid using around flames and sources of ignition. Keep out of reach of children. The wipe will start to break down in the presence of water – e.g. after showering. If eye contact / ingestion occur, seek immediate medical attention. Reuse of this product may affect the performance and create additional risks to the user. Use only as directed.

Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina etc.). If in doubt, patch test a small area prior to use. Discontinue use if irritation occurs. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. The wipes are individually packaged for single use only. Do not use if the packaging is damaged or opened.

## Instructions For Use

1. Skin should be clean and dry prior to application.
2. To open the sachet, tear along either notch across the top.
3. Remove the wipe from the sachet and gently wipe over the intended area.
4. LBF Hydrosolve Barrier Film dries quickly. Once dried, it will not affect any further adhesion of appliances.
5. Place the used wipe in the toilet bowl and flush away. Dispose of the outer sachet in your domestic rubbish bin.

## Ingredients

Siloxane copolymers

## Storage and Shelf Life

Store away from direct sunlight. Store at ambient temperature and humidity. For shelf life, refer to the expiry date on each sachet.



Latex  
Free



## Ordering Information

Product Code	Description	No. per box
3828	LBF Hydrosolve Flushable Barrier Film Wipes	30
3506	Appeel Hydrosolve Flushable Medical Adhesive Remover Wipes	30

Last Revised August 2014

 CliniMed Ltd., Cavell House,  
Knaves Beech Way, Loudwater,  
High Wycombe, Bucks HP10 9QY  
Tel 01628 850100 Fax 01628 527312  
[www.clinimed.co.uk](http://www.clinimed.co.uk)

CliniMed® and LBF® are registered  
trademarks of CliniMed (Holdings) Ltd.

**CliniMed**®  
Essentials ●●●●●