ENGLISH (GB)

DEVICE DESCRIPTION:
Device Name: iPRO ELITE - Silicone Eye Protector
<p>Designed to reduce the risk of corneal abrasions & other common hazards encountered during anaesthesia.</p>
<p>The eye protector helps provide a consistent and even pressure on the eyelid. By keeping the eyelid shut, this product helps maintain the natural tear film on the cornea during the procedure.</p>
INDICATIONS FOR USE:
A mild adhesive eye protector, to be used as an alternative to adhesive tape during surgery.
CONTRAINDICATIONS:
There are no known Contraindications for this device.
WARNINGS & PRECAUTIONS:
The device is only to be used by suitably qualified clinicians, who are experienced with the specific techniques required for these medical devices.
This device is single-use. Do not re-use.
CAUTION:
Use tabs only for placement and removal of the device. Take care upon removal of the device from the patient’s eye area.
INSTRUCTIONS FOR USE
Illustration A
- Remove the iPro pad from the backing paper, using the coloured tab.
Illustration B
- Holding the coloured tab, position the iPro over the eye.
- Ensure the eyelid is closed and fully covered.
Illustration C
- Press gently to ensure the eye protector is securely in place.
- Repeat procedure for the other eye.

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