

LBF Barrier Cream (2g sachet)

Instructions For Use

Indications: LBF Barrier Cream is indicated for the protection of intact skin that is at risk or likely to be compromised by corrosive bodily fluids (including urine and faeces) or adhesives.

Contraindications: LBF Barrier Cream should not be used on broken or infected skin. The cream should not be used in cases where there is a known allergic sensitivity to any of the ingredients.

Instructions for use:

Stoma Care: Remove any urine or faecal matter, cleanse and dry skin, apply LBF Barrier Cream sparingly around peristomal skin as required and rub in well before applying the pouch. Repeat application as required.

Continence Care: Remove any urine or faecal matter, cleanse and dry skin, apply LBF Barrier Cream around the area as required. Repeat application after each incontinent episode or as required.

Ingredients: Purified Water, Disodium EDTA, Methyl Parabens, Zinc Oxide, Sodium Chloride, Propylene Glycol, Petrolatum, Dimethicone, Cetyl Dimethicone, Cetyl PEG/PPG-10/1 Dimethicone, Isopropyl Palmitate, Propyl Parabens, Beeswax and Hydrogenated Castor Oil.

Cautions: External use only. If the skin condition changes stop use of product and contact a physician. Do not use near eyes or ears. Do not use if the sachet is damaged. Keep out of reach of children. Use only as directed. Check expiry date before use. Dispose of safely after use.

Storage: Store away from direct sunlight and humidity.



CliniMed®
Essentials ●●●●●

CliniMed® and LBF® are registered trademarks of CliniMed (Holdings) Ltd.

■ CliniMed Ltd., Cavell House, Knaves Beech Way, Loudwater, High Wycombe, Bucks HP10 9QY

Tel: 01628 850100

Fax: 01628 527312

Email: enquiries@clinimed.co.uk

Web: www.clinimed.co.uk

