

LoFric[®]
Hydro-Kit™

All you need



www.lofric.com



LoFric Hydro-Kit. The all-in-one catheter system that's ready when you are.

- **Any time, any place**
 - small and convenient
- **Hassle-free**
 - easy to handle and use wherever you are
- **Proven performance**
 - safe and well documented

Any time, any place

Having to use a catheter may make your daily activities more challenging, but it certainly shouldn't stop anyone either. LoFric[®] Hydro-Kit[™] is the all-in-one kit that combines collection bag, catheter tube and water, making it easier to self-catheterise whenever you're ready, wherever you are.


LoFric Hydro-Kit is small. It's flexible and easy to handle. And it's all-in-one. No preparations, no planning.

LoFric Hydro-Kit has its own water. You don't need to touch it with your fingers; a handling guide ensures you can keep a firm grip on the hydrophilic catheter. And the collection bag means it can be used anywhere, even if there's no toilet.


Getting out more often can be challenging for any number of reasons. With LoFric Hydro-Kit, self-catheterisation isn't one of them.

LoFric Hydro-Kit

For men






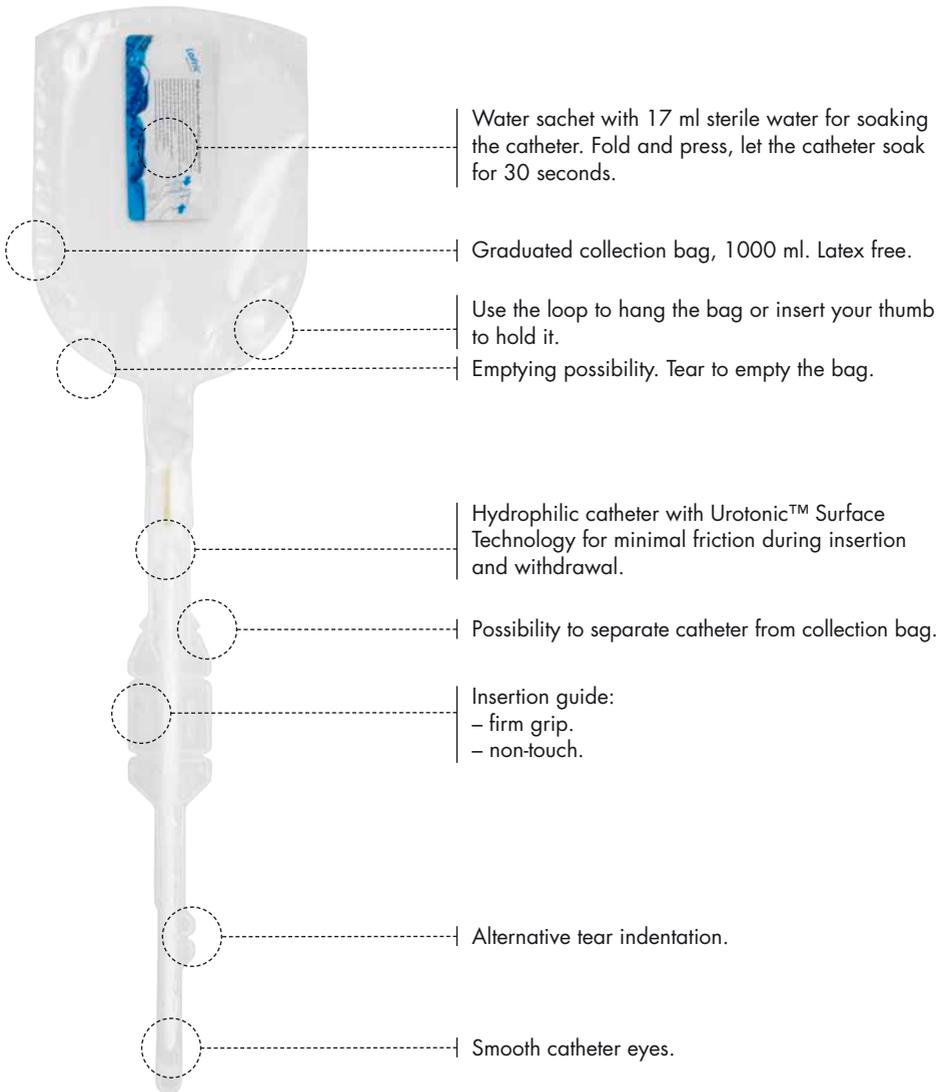


For women and children

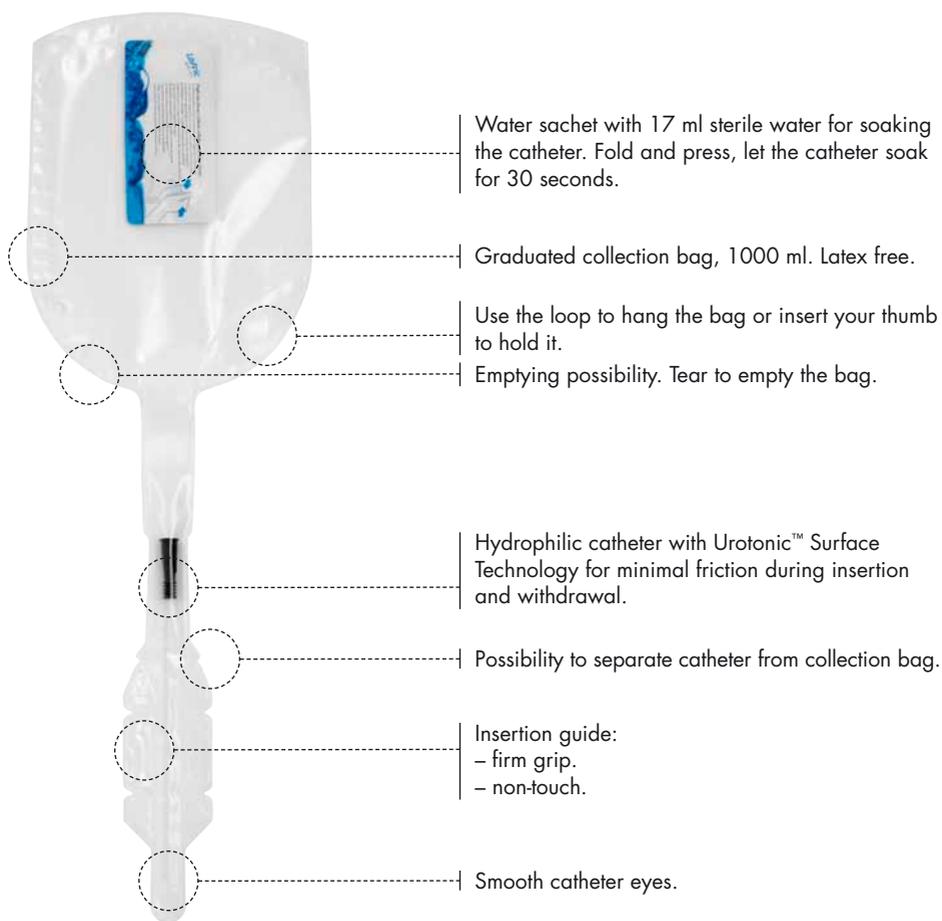



How to use LoFric Hydro-Kit

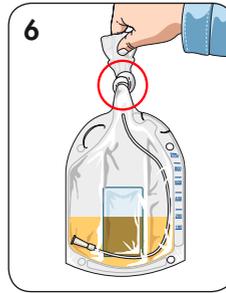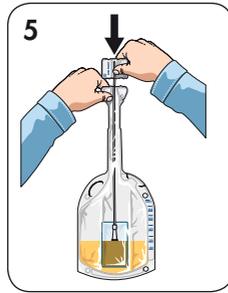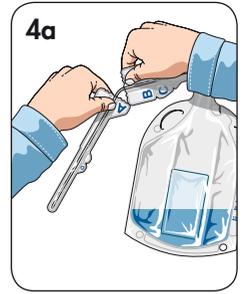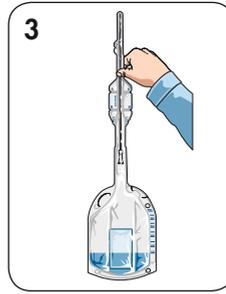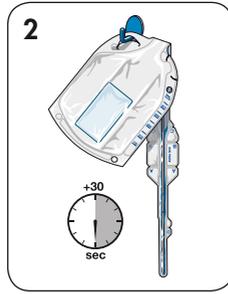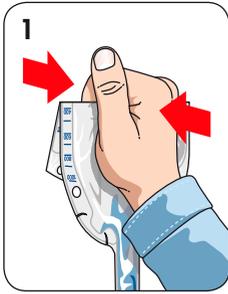

1. Hold the product upright. Fold the water pocket and squeeze. Let the water run down to the catheter.
2. Soak the catheter for 30 seconds while you prepare yourself.
3. Turn the product upside down to allow the water to drain into the collection bag.
4. Tear at indentation "A" to uncover the tip of the catheter. Tear at indentation "B". Use section "B" as a handling aid. Catheterise.
5. Push the catheter back into the collection bag.
6. Tie a knot to seal the bag.
7. Empty the collection bag by tearing at the indentation.

The water pocket contains sterile water that should only be used for soaking the LoFric catheter.

Hint: To ensure a tight seal between the catheter and the collection bag before catheterisation: gently pull the catheter out of the packaging until the connector comes to a full stop.

Hint: After catheterisation, you can throw away the collection bag and the catheter separately by tearing at indentation "C".

NB. *LoFric Hydro-Kit is for single use only.*



Lower friction for higher comfort

The LoFric Hydro-Kit catheter, like all LoFric catheters, is coated with Urotonic Surface Technology. This hydrophilic coating binds water, creating a very wet and slippery surface that is as frictionless as a catheter tube can be – both for insertion and withdrawal.

LoFric are the only catheters backed by long-term documentation. The Urotonic Surface Technology is fully proven and documented. For more than 25 years the technology has ensured the lowest possible friction, and therefore minimised health complications. The result is this assurance: LoFric Hydro-Kit has 90–95 % lower friction than traditional catheters which are used in combination with gel.



LoFric Hydro-Kit – product range

| Reference. | Type | Length | Size |
|------------|---------------------------------------|--------|------|
| 983 08 00 | LoFric® Hydro-Kit™ Nelaton | 40 cm | CH08 |
| 983 10 00 | LoFric® Hydro-Kit™ Nelaton | 40 cm | CH10 |
| 983 12 00 | LoFric® Hydro-Kit™ Nelaton | 40 cm | CH12 |
| 983 14 00 | LoFric® Hydro-Kit™ Nelaton | 40 cm | CH14 |
| 983 16 00 | LoFric® Hydro-Kit™ Nelaton | 40 cm | CH16 |
| 983 18 00 | LoFric® Hydro-Kit™ Nelaton | 40 cm | CH18 |
| 987 10 00 | LoFric® Hydro-Kit™ Tiemann | 40 cm | CH10 |
| 987 12 00 | LoFric® Hydro-Kit™ Tiemann | 40 cm | CH12 |
| 987 14 00 | LoFric® Hydro-Kit™ Tiemann | 40 cm | CH14 |
| 987 16 00 | LoFric® Hydro-Kit™ Tiemann | 40 cm | CH16 |
| 987 18 00 | LoFric® Hydro-Kit™ Tiemann | 40 cm | CH18 |
| 423 08 00 | LoFric® Hydro-Kit™ Nelaton | 20 cm | CH08 |
| 423 10 00 | LoFric® Hydro-Kit™ Nelaton | 20 cm | CH10 |
| 423 12 00 | LoFric® Hydro-Kit™ Nelaton | 20 cm | CH12 |
| 423 14 00 | LoFric® Hydro-Kit™ Nelaton | 20 cm | CH14 |
| 423 16 00 | LoFric® Hydro-Kit™ Nelaton | 20 cm | CH16 |
| 423 18 00 | LoFric® Hydro-Kit™ Nelaton | 20 cm | CH18 |
| 421 06 00 | LoFric® Hydro-Kit™ Nelaton Paediatric | 20 cm | CH06 |
| 421 08 00 | LoFric® Hydro-Kit™ Nelaton Paediatric | 20 cm | CH08 |
| 421 10 00 | LoFric® Hydro-Kit™ Nelaton Paediatric | 20 cm | CH10 |

Wide range of LoFric products

There are many types of LoFric products with Urotonic Surface Technology; each of them fulfils a particular need.

You can read more about them at www.lofric.com





Knowledge, education and support

Do you need product information or help with therapeutic questions? We have comprehensive knowledge in all the areas associated with catheterisation, dilatation and instillation. We are happy to act as a resource, or to provide education.

When it comes to education and information, we can offer a broad programme – both for health care staff and users. You may use our training staff, or receive training and information on video, CD and in print. Apart from that, we can supply information on products, clinical studies, user guides, starter packs and much more.

Contact us for all your needs.

www.lofric.com

Wellspect HealthCare

Aminogatan 1, P.O. Box 14, SE-431 21 Mölndal, Sweden.
Phone: +46 31 376 40 00. www.lofric.com

wellspect
HEALTHCARE