


Melgisorb® Plus

Absorbent alginate dressing

Melgisorb Plus is a highly absorbent dressing for a wide range of moderately to heavily exuding wounds.

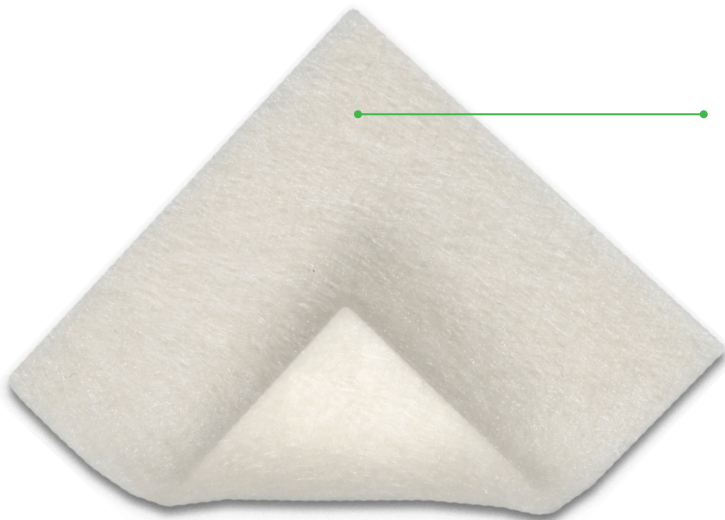


Hydrophilic alginate fibers


Soft sterile dressing made of Calcium Sodium Alginate fibers

- Excellent absorption - for high exuding wounds
- High wet strength - to allow removal in one piece
- High retention - to limit maceration

How to use Melgisorb® Plus

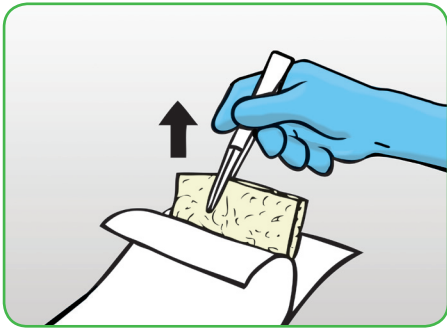


1. Select a size of Melgisorb Plus that is slightly larger than the wound. Open the package by pulling.

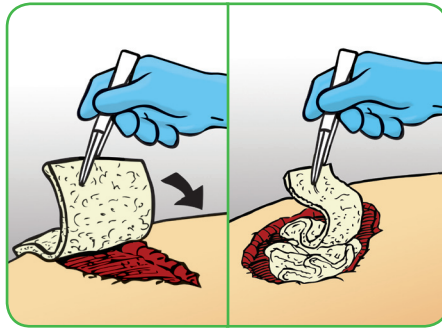

2. Apply a dry Melgisorb Plus product to the moist wound bed.

Shallow wounds: Choose the correct size of flat dressing to be able to cover the entire wound.

Deep wounds and cavities: Cut the appropriate length of the cavity dressing and pack the wound loosely.

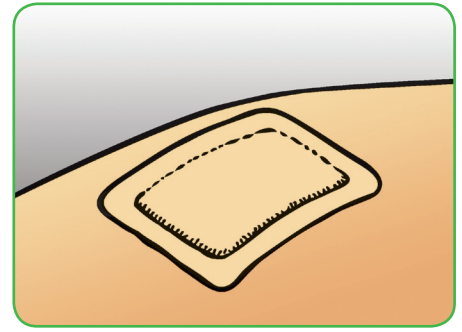

3. Cover and secure Melgisorb Plus with an appropriate secondary dressing such as Meplix® Border.

Compression therapy may be used in conjunction with Melgisorb Plus.

How Melgisorb Plus works

Melgisorb Plus absorbs large amounts of wound exudate. When absorbing, the alginate fibres form a soft moist gel, giving a moist environment conducive to wound healing. The gel can easily be removed by irrigating with 0.9% saline.

Benefits of Melgisorb Plus

- Highly absorbent
- Does not adhere to the wounds site when gelled
- Suitable for cavity wounds
- Creates a gel on contact with exudate or solutions containing sodium ions (saline)
- Soft/ pliable/ conformable
- Can be cut to shape
- Non-sensitising

Precautions

Melgisorb Plus is not indicated for dry wounds, third degree burns or surgical implantations.

If infection is suspected, follow local routines given by health care professionals

If reused performance of the product may deteriorate, cross-contamination may occur.

Intended Use

Melgisorb Plus is intended for a wide range of moderately to heavily exuding wounds, both infected and noninfected, such as:

- Pressure sores
- Venous and arterial ulcers
- Diabetic ulcers
- Donor sites
- Post operative wounds
- Dermal lesions and other external wounds inflicted by trauma

Configurations available both for flat and cavity wounds

Melgisorb Plus (Sterile packed)

Art. no	Size cm	Size inch	Pcs/shelf cont.	Pcs/transp. cont.
252000	5x5	2x2	10	100
252200	10x10	4x4	10	100
252500	10x20	4x8	10	100
253500	3x45	1.2x17.7	5	50

Note: Not all articles are available in every country. Please contact your local Mölnlycke Health Care representative for information about articles available in your country.



Find out more at www.molnlycke.com

Mölnlycke Health Care AB, Box 13080, SE-40252 Göteborg, Sweden
Phone +46317223000

The Mölnlycke Health Care, Mepilex® Border names and respective logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies.

© 2013 Mölnlycke Health Care AB. All rights reserved.

