



Product information

Indications

Navina™ Systems from Wellspect HealthCare are developed to help adults who suffer from fecal incontinence, chronic constipation, and/or time consuming bowel management. The user may be wheel chair bound and/or have reduced motoric hand function. By instilling water up into the lower part of the colon, the peristaltic muscles in the bowel can be triggered and start to evacuate the colon and rectum.

Contraindications




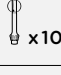



You should not use Navina™ Classic system if you have one or more of the following:

- Known anal or colorectal stenosis
- Active inflammatory bowel disease.
- Acute diverticulitis.
- Colorectal cancer.
- Ischemic colitis.

- Within three months of anal or colorectal surgery.
- You are within 4 weeks of previous endoscopic polypectomy.

As the list of contraindications may not be exhaustive, healthcare professionals will always consider individual user factors as well.

Navina Classic system

Reference	Type	Content	Life time
69005	Navina Classic system 	1 Navina Classic control unit 1 Water container 1 Tube set 2 Rectal catheters 1 Navina case 1 Accessory set	
69006	Navina Classic system regular - refill 	1 Navina Classic control unit 1 Water container 1 Tube set 2 Rectal catheters	
69003	Consumable set regular 	1 Water container 15 Rectal catheters	15 uses Single use only
68940	Rectal catheter set 	10 Rectal catheters	Single use only
69011	Tube set 	1 Catheter tube 1 Water container tube	100 uses 100 uses
69012	Accessory set 	2 Grip rings (1 vertical, 1 horizontal) 1 Lanyard 1 Positioning strap 1 Navina Smart positioning clip	
69013	Navina Classic control unit 	1 Navina Classic control unit	100 uses

All Navina Classic components should be disposed of as household waste.

Navina™ Classic

WHEN THE BOWEL WORKS, LIFE WORKS

1. Tubing

- Easy to grip and handle.
- Color-coded and secure connectors to ensure correct assembly intuitively.
- Easy to clean.



2. Water container

- Small collapsible water container for discreet storage and transport.
- Sturdy and with an integrated scale that maintains its accuracy over time.
- Single compartment design makes it easy to fill and empty.
- Easy to open and close even with reduced hand strength and function.
- Integrated lid (no assembly needed) with pressure valve for increased safety.

3. Disposable catheter

- Disposable hydrophilic catheter with rounded tip for maximum comfort, hygiene and safety.
- Flexible balloon design seals the rectum to ensure catheter stays in place and provides a good seal throughout treatment.
- Good grip with connector makes handling easier.
- Balloons tested individually for optimal function and to ensure a good safety margin between maximum inflation and bursting point.

4. Manual pump

- Color-coded water and air inflation pumps for clear understanding.
- Easy pumping, with or without unit strapped to leg.
- Large, intuitive controls.

wellspect.com

