

User-friendly handle for good grip and non-touch technique. Connects to a standard collection bag.

Urotonic<sup>™</sup> Surface Technology for low friction and maximal comfort.

Smooth catheter eyes.

Optimal tube length, ensures both catheter eyes are in the bladder for complete bladder emptying.

Free from PVC, phthalates and latex.



User friendly dispenser box.

## Intended Use

The LoFric<sup>®</sup> Sense<sup>™</sup> single use urinary catheter is intended for intermittent urinary catheterisation.

## Sterilisation & Shelf Life

All catheters are sterilised by irradiation. Information about the recommended best-before date is printed on the package.

## Package

Individual "peel-open" packaging.  
30 catheters per carton.  
4 cartons per shipping box.

## Material Composition

**Hydrophilic surface coating:** Polyvinyl pyrrolidone (PVP).

**Catheter:** polyolefin-based elastomer (POBE).

**Wetting pouch:** Laminate foil of polyethylene terephthalate (PET), aluminum and polyethylene (PE). Sterile water with sodium chloride (NaCl).

**Item package:** Laminate foil of polyester with aluminium oxide (PET/Alox), linear low density polyethylene (LLDPE), aluminium foil.

## Product Range

Reference	Type	Length	Size
41608XX*	LoFric <sup>®</sup> Sense <sup>™</sup>	15 cm	CH 08
41610XX*	LoFric <sup>®</sup> Sense <sup>™</sup>	15 cm	CH 10
41612XX*	LoFric <sup>®</sup> Sense <sup>™</sup>	15 cm	CH 12
41614XX*	LoFric <sup>®</sup> Sense <sup>™</sup>	15 cm	CH 14


\* Country code ref. nr

- **Safe** – comfortable, complete emptying and proven safe for long-term use.
- **Hygienic** – before, during and after use.
- **Convenient** – functional design from storage to disposal.


**Smarter design, safer catheterisation.  
Now that makes sense.**




# Instructions

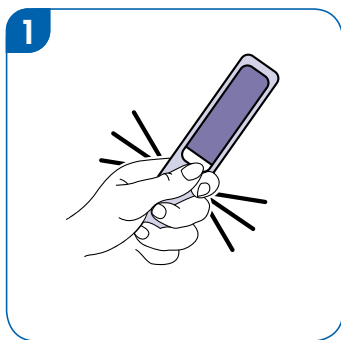


**1.** Squeeze and break the water packet to soak the catheter before use.

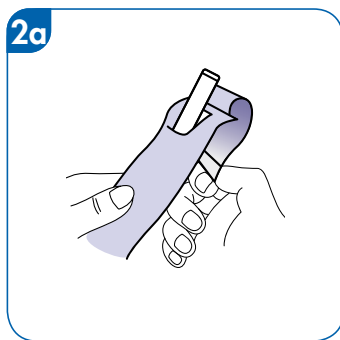

**2a.** Peel open.

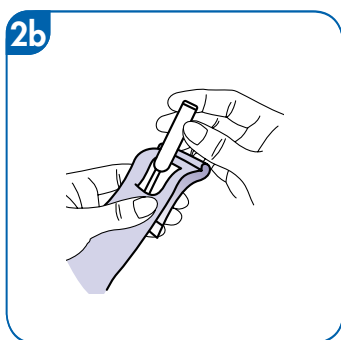

**2b.** Hold flap in place and remove catheter.

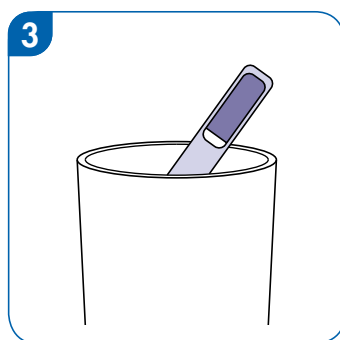

**3.** The outer packaging doubles as a hygienic and discreet disposal pouch.