


### MEASURING GUIDE


#### BELOW KNEE

Size	Calf(C)	Lenght(L1)	Product Code	NHS SC Code
Small Regular	28-32cm	<42cm	221	EGD7429
Small Long	28-32cm	≥42cm	231	EGD7430
Medium Regular	32-36cm	<42cm	222	EGD7431
Medium Long	32-36cm	≥42cm	232	EGD7432
Large Regular	36-40cm	<42cm	223	EGD7433
Large Long	36-40cm	≥42cm	233	EGD7434
X-Large Regular	40-44cm	<42cm	224	EGD7435
X-Large Long	40-44cm	≥42cm	234	EGD7436
XX-Large Regular	44-50cm	<42cm	225	EGD7437
XX-Large Long	44-50cm	≥42cm	235	EGD7438
XXX-Large Regular	50-56cm	<42cm	226	EGD7451
XXX-Large Long	50-56cm	≥42cm	236	EGD7452
XXXX-Large Reg	56-62cm	<42cm	227	TBC
XXXX-Large Long	56-62cm	≥42cm	237	TBC
XXXXX-Large Reg	62-68cm	<42cm	228	TBC
XXXXX-Large Reg	62-68cm	≥42cm	238	TBC

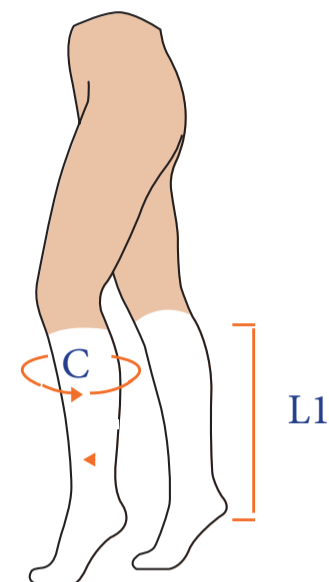


#### Thigh Length

Size	Calf(C)	Thigh(T)	Lenght(L2)	Product Code	NHS SC Code
Small Small	28-32cm	<65cm	<74cm	241	EGD8474
Small Regular	28-32cm	<65cm	74-80cm	251	EGD7440
Small Long	28-32cm	<65cm	>80cm	261	EGD8473
Medium Small	32-36cm	<65cm	<74cm	242	EGD7442
Medium Regular	32-36cm	<65cm	74-80cm	252	EGD7443
Medium Long	32-36cm	<65cm	>80cm	262	EGD7444
Large Small	36-40cm	<65cm	<74cm	243	EGD7445
Large Regular	36-40cm	<65cm	74-80cm	253	EGD7446
Large Long	36-40cm	<65cm	>80cm	263	EGD7447
X-Large Small	40-44cm	65-81cm	<74cm	244	EGD7448
X-Large Regular	40-44cm	65-81cm	74-80cm	254	EGD7449
X-Large Long	40-44cm	65-81cm	>80cm	264	EGD7472

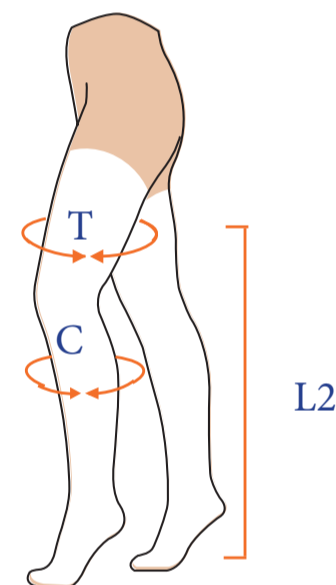

#### Thigh Length with Belt

Size	Calf(C)	Thigh(T)	Lenght(L2)	Product Code	NHS SC Code
Small Regular	28cm	<65cm	74-80cm	281	EGD8484
Small Long	28cm	<65cm	>80cm	291	EGD8488
Medium Regular	32cm	<65cm	74-80cm	282	EGD8485
Medium Long	32cm	<65cm	>80cm	292	EGD8489
Large Regular	36cm	<65cm	74-80cm	283	EGD8486
Large Long	36cm	<65cm	>80cm	283	EGD8490
X-Large Regular	40cm	65-81cm	74-80cm	284	EGD8487
X-Large Long	40cm	65-81cm	>80cm	294	EGD8491

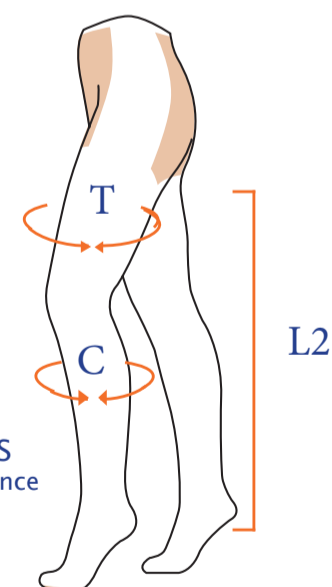

#### Prevenex® Application Guidelines



1. Insert hand into stocking as far as the heel area. Grasp heel section and turn stocking inside out as far as the heel
2. Insert the foot into the stocking, ensuring the patient's heel is in the blue heel area
3. Gently ease the stocking up the leg working in small sections, to ensure even pressure application across the whole leg
4. Check that the toe inspection opening is correctly positioned below ball of foot and stocking feels comfortable

Stockings should be removed for 30 minutes each day for hygiene and skin care purposes (NICE Jan 2010). Recheck patient measurements every three days and if different from previous measurement, provide new size of Preventex Stockings accordingly.

#### BELOW KNEE STOCKINGS

- Measure the calf circumference at the widest point (C)
- Measure the length from behind the knee to the bottom of the heel. (L1)

#### THIGH LENGTH STOCKINGS

- Measure the length from the buttock fold to the bottom of the heel. (L2)
- If thigh circumference is over 81 cm select below knee style only.