

Appeel[®] Hydrosolve Flushable Medical Adhesive Remover Wipes

Product Description

Appeel Hydrosolve Flushable Medical Adhesive Remover Wipes are flushable wipes containing a non-stinging solvent, which aids the removal of adhesive appliances. They are fast-drying, leaving no residue on the skin, which allows adhesive appliances to be re-applied shortly after use. The wipe is made using a dispersible fabric which disintegrates in seconds when placed in water. The wipe can be disposed of in the toilet bowl and flushed away.

Indications

Appeel Hydrosolve Flushable Medical Adhesive Remover Wipes reduce pain, skin stripping and trauma that can be caused by the removal of adhesive based products such as tapes, dressings, stoma pouches and flanges. The product may be used on adults, children and infants. For external use only.

Contraindications

Do not use on mucosal membranes and broken skin. It is not to be used on infected skin or in cases where there is a known sensitivity to any of the ingredients.

Caution

Appeel Hydrosolve Flushable Medical Adhesive Remover, in liquid form is flammable; use in well a ventilated area. Avoid using around flames and sources of ignition. Keep out of reach of children. The wipe will start to break down in the presence of water – e.g. after showering. If eye contact / ingestion occur, seek immediate medical attention. Reuse of this product may affect the performance and create additional risks to the user. Use only as directed.

Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina etc.). If in doubt, patch test a small area prior to use. Discontinue use if irritation occurs. Should redness or other signs of irritation appear, or irritation persists, consult a medical professional. The wipes are individually packaged for single use only. Do not use if the packaging is damaged or opened.

Instructions For Use:

1. Skin should be clean and dry prior to application.
2. To open the sachet, tear along either notch across the top.
3. Remove the wipe from the sachet and gently wipe over a permeable tape/dressing/stoma appliance or under the edges of an impermeable tape/dressing/stoma appliance to release it from the skin.
4. Appeel Hydrosolve Flushable Adhesive Remover dries quickly, leaving no residue. Once dried, it will not affect any further adhesion of appliances.
5. Place the used wipe in the toilet bowl and flush away. Dispose of the outer sachet in your domestic rubbish bin.

Ingredients

Siloxanes.

Storage and Shelf Life

Store away from direct sunlight. Store at ambient temperature and humidity. For shelf life, refer to the expiry date on each sachet.



Latex
Free



Ordering Information

Product Code	Description	No. per box
3506	Appeel Hydrosolve Flushable Medical Adhesive Remover Wipes	30
3828	LBF Hydrosolve Flushable Barrier Film Wipes	30

Last Revised August 2014

 CliniMed Ltd., Cavell House,
Knaves Beech Way, Loudwater,
High Wycombe, Bucks HP10 9QY
Tel 01628 850100 Fax 01628 527312
www.clinimed.co.uk

CliniMed[®]
Essentials ●●●●●

CliniMed[®] and Appeel[®] are registered
trademarks of CliniMed (Holdings) Ltd.

PID 934