

CBC-VECOS®


Single-use tourniquet

Hygienic. Gentle. Easy. Safe.




CBC-VECOS® = VEIN


COMPRESSION
SYSTEM

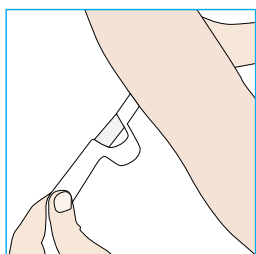


1. Place **white side** on the patient's arm.

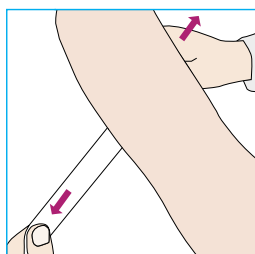

2. Build up compression force by extending.

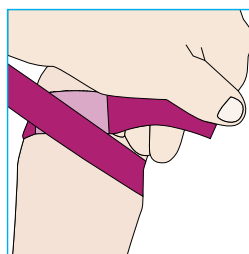

3. Place strap tightly around the arm and attach in the velcroed area.

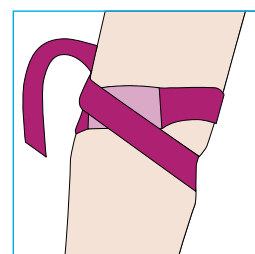

4. Simply pull to release the strap.

Do not use again after use and dispose of in accordance with regulations.

Watch the video to see how to apply it:



Hygienic

- CBC-VECOS® protects against the transmission of germs and its single use prevents cross-contamination.
- CBC-VECOS® reduces the risk of hospital-acquired infections such as MRSA and fosters hygiene standards for the medical staff.
- No complex cleaning or disinfection of the tourniquet.

Gentle

- The velvety soft fleece is gentle on the skin providing the best possible comfort for the patient.
- No bruising, lacing or constricting of the skin.
- Standard: 3 cm wide, 50 cm long. For obese patients, we recommend: 3.5 cm wide, 65 cm long.

Easy

- CBC-VECOS® is easy to apply. The micro-velcro®* allows for easy opening and closing.
- The micro-velcroed strip attaches securely to any stretch of the strap, allowing for easy re-tightening if needed.

*Velcro® is a registered trademark of Velcro Industries B.V.

Safe

- CBC-VECOS® helps to build up the optimal compression force. The elastic strap defines the correct minimum compression force reliably, ensuring optimal occlusion.
- CBC-VECOS® is free from latex, silicone and plasticiser.

Ordering details

Product Description	CBC-VECOS® Tourniquet blackberry 30 mm
Strap:	3 cm wide, 50 cm long
REF 521001 V 125	125 pcs. in dispenser box
REF 521001 B 125	125 pieces in refill pack

For obese patients, we recommend,

Product Description	CBC-VECOS® Tourniquet blackberry 35 mm
Strap:	3.5 cm wide, 65 cm long
REF 521002 V 100	100 pcs. in dispenser box
REF 521002 B 100	100 pieces in refill pack



Fannin

Caring for Life since 1829

www.fannin.eu

Fannin (UK) Ltd.
Westminster Industrial Estate,
Repton Road, Measham,
Swadlincote, Derbyshire,
DE12 7DT
T: +44 (0) 1530 514566
E: serviceuk@fannin.eu

