# Flaminal <sup>®</sup>Forte SIMPLICITY IN TREATMENT



### MODERATELY TO HEAVILY EXUDING WOUNDS

# Flaminal <sup>®</sup>Hydro SIMPLICITY IN TREATMENT



### **SLIGHTLY TO MODERATELY EXUDING WOUNDS**





### INDICATIONS

#### Flaminal<sup>®</sup> Hydro

- skin tears
- complex grazes
- partial thickness burns
- ulcers (venous, arterial, diabetic)
- radiodermatitis
- oncological wounds
- surgical wounds
- pressure ulcers

## **HOW TO USE**

Renewing Flaminal<sup>®</sup> Hydro/Forte on a wound is easy and painless:

- 1. First clean and rinse the wound with Flamirins®, dab dry.
- Cover the entire wound bed with a sufficiently thick layer (approx. 0.5 cm) of Flaminal® Hydro/ Forte.
- 3. Cover with a paraffin gauze or a non-adhesive dressing.



1, 2, 3 = HEALED

#### Flaminal® Forte

- partial thickness burns
- ulcers (venous, diabetic)
- pressure ulcers
- oncological wounds

## GOOD TO KNOW

- The choice of Flaminal® Hydro/Forte is based on the amount of exudate. If in doubt, start with Flaminal® Hydro.
- During the initial phase of the treatment the wound may become larger due to debridement.
- During Flaminal® Hydro/Forte treatment, dry whitish alginate flakes may appear on the wound edge. They should not be removed.
- The dressing with Flaminal® Hydro/Forte can remain in place as long as the gel structure is intact (change every 2 to 3 days or daily for necrotic, fibrinous wounds).



Not all presentations are available in every country.

