

ONLY

ONE

FL  **W** **SAFE**®
DISPOSABLE CPAP SYSTEM

Flow-Safe Features:

- 4 Mask Styles
- Built-in Manometer
- Built-in Pressure Relief Valve
- Titratable CPAP Pressure (no separate fixed CPAP valves)
- Accepts Standard Nebulizers
- Highly Portable, Easy Set-up
- Latex-Free

Flow-Safe CPAP Mask Descriptions:

Deluxe Mask with . . .

STRAIGHT ROTATING PORT.

This Flow-Safe mask style is ideal for EMS. The netted headgear fabric is cool and easily stretches when placing on patients. The velcro straps allow easy adjustment for proper fit. Soft cushion tabs on the forehead decrease pressure on the patient's nose.

This deluxe CPAP mask port style accepts a standard nebulizer in-line and is key to agencies that have added asthma patients to their CPAP protocols.

Should the patient require medication delivery after the mask has been placed, the mask easily lifts up and/or the blue quick connect clips can be removed providing easy access for delivering nitroglycerin or other meds. — *Available with or without oxygen ports.*



Deluxe Mask with . . .

90° SWIVEL PORT AND ANTI-ASPHYXIA VALVE.

While some EMS agencies like this mask style, it is well suited for hospital use where they may prefer the anti-asphyxia valve for their CPAP patients. The Mercury Flow-Safe CPAP system is an open system so asphyxia is not a concern.

It includes the same headgear fabric that stretches and is easy to place on patients and the velcro straps easily adjust to fit. Soft cushion tabs on the forehead decrease pressure on the patient's nose.

This deluxe CPAP mask port style accepts a standard nebulizer in-line. Due to the swivel feature, this mask will require an additional adapter to maintain the nebulizer in the upright position.

Should the patient require medication delivery after the mask has been placed, the mask easily lifts up and/or the blue quick connect clips can be removed providing easy access to the patient's mouth.

— *Available with or without oxygen ports.*



www.mercurymed.com

MercuryMedical®

Your Need ... Our Innovation®

ONLY ONE

Flow-Safe Features:

- 4 Mask Styles
- Built-in Manometer
- Built-in Pressure Relief Valve
- Titratable CPAP Pressure (no separate fixed CPAP valves)
- Accepts Standard Nebulizers
- Highly Portable, Easy Set-up
- Latex-Free

FLOW**SAFE**[®]
DISPOSABLE CPAP SYSTEM

Flow-Safe CPAP Mask Descriptions:

Intermediate Cushion Mask with . . . FABRIC HEADSTRAP.

Similar to the original cushion mask, this mask also includes an inflation valve. It features a comfortable, not-stretchy fabric material headstrap, in place of the blue headstrap, offering an affordable option. This mask also features quick connect clips to the cushion mask allowing for easier access for administering medication after CPAP therapy is implemented.

NEW



Cushion Mask with . . . BLUE NEOPRENE HEADSTRAP.

The original cushion mask with blue headstrap is available when cost is a consideration.

Standard nebulizers connect in-line.

The cushion mask contains a color-coded hook ring for attaching the blue neoprene headstrap.

If the patient requires medication after CPAP therapy has begun, they can detach the blue headstrap to access the oral cavity and re-attach the straps to the hook rings to continue CPAP.



11300 - 49th Street North • Clearwater, Florida 33762-4807 USA
Tel. (800) 237-6418 / (727) 573-0088 • Fax (800) 990-6375

8/2012

ML2011-191

Rev. B

www.mercurymed.com

MercuryMedical[®]

Your Need ... Our Innovation[®]