

**LBF® Sterile No Sting Barrier Film Foam Applicator - 1ml**

**Product Description**

LBF Sterile No Sting Barrier Film is applied as a non-stinging solvent, which dries rapidly forming a transparent protective film. LBF Sterile No Sting Barrier Film is indicated for the protection of intact or damaged skin that is at risk or likely to be compromised by corrosive bodily fluids (including urine, faeces, digestive enzymes), wound drainage, adhesives and friction.

**Indications**

LBF Sterile No Sting Barrier Film is a liquid intended for external use as a film-forming product. When applied to the skin, it forms a protective barrier between the skin and corrosive bodily fluids, adhesive products and friction. It is intended as a primary barrier against irritation from bodily fluids. The product may be used on adults, children and infants over 12 months of age.

**Contraindications**

LBF Sterile No Sting Barrier Film is not to be used as the only covering in situations that require dressing protection from bacterial contamination / penetration, e.g. intravenous therapy catheter sites and full or partial thickness wounds. It is not to be used on infected skin or in cases where there is a known sensitivity to any of the ingredients.

**Warning**

LBF Sterile No Sting Barrier Film, in liquid form, is flammable; use in well ventilated area. Avoid using around flames and sources of ignition. Keep out of reach of children. Product is sterile unless open or damaged. LBF Sterile No Sting Barrier Film is for external use only. Reuse of this product may affect the performance and create potential additional risks to the user. If eye contact / ingestion occur seek immediate medical attention. Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina etc.).

**Precautions**

1. Should redness or other signs of irritation appear, or if redness or signs of irritation persist, consult a medical professional.
2. Use of other barrier products, ointments, creams or lotions may significantly reduce the effectiveness of LBF Sterile No Sting Barrier Film.
3. The foam applicators are individually packaged for single use only. Do not use if the packaging is damaged or opened.

**Instructions For Use**

1. Skin should be clean and dry prior to application of LBF Sterile No Sting Barrier Film.
2. Apply a uniform coating of film to the area of concern using the foam applicator. Do not apply directly onto the wound.
3. If an area of skin is missed, reapply to that area only after the first application of LBF Sterile No Sting Barrier Film has dried (approximately 30 seconds).
4. If LBF Sterile No Sting Barrier Film is applied to an area with skin folds or other skin-to-skin contact, make sure that skin-contact areas are separated to allow them to dry before returning to normal position.

1597/0612/1

205mm

5. When used under adhesives tapes, dressings or devices:
  - a). Allow LBF Sterile No Sting Barrier Film to dry before covering with dressings or adhesive products.
  - b). Reapplication of LBF Sterile No Sting Barrier Film is necessary each time dressing and / or adhesive products are changed; the barrier is removed by the adhesive.
6. When used as a protectant against bodily fluids, faeces or urine and no adhesive products are applied to the skin:
  - a). Reapplication of LBF Sterile No Sting Barrier Film is recommended every 24-48 hours, depending on the frequency of cleaning.
  - b). In extreme cases (e.g. constant diarrheal stooling) with very frequent cleansing, more frequent applications may be necessary (i.e. every 12-24 hours).
7. If desired, the film can be removed by using most medical adhesive removers as directed. Clean and dry the involved area and reapply LBF Sterile No Sting Barrier Film.

**Ingredients**

Siloxane copolymers.

**Storage and Shelf Life**

Store away from direct sunlight, ambient temperature and humidity. For shelf life, refer to the expiry date on each sachet.





**Ordering Information**

Product Code	Description	No. per box
3824	LBF Sterile No Sting Barrier Film Foam Applicator, 1ml	5

CliniMed Ltd., Cavell House,  
Knaves Beech Way, Loudwater,  
High Wycombe, Bucks HP10 9QY  
Tel 01628 850100 Fax 01628 527312  
www.clinimed.co.uk

CliniMed® and LBF® are registered trademarks of CliniMed (Holdings) Ltd.

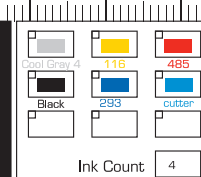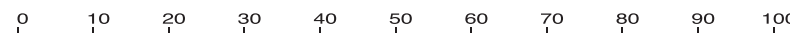

Client	Flexible Medical	Date	22-November-12
Job Number	66106	Step Size	
Order Number		Operator	sarah.r
Job Title	LBF Leaflet		

approved \_\_\_\_\_ date \_\_\_\_\_

approved subject to \_\_\_\_\_

\_\_\_\_\_ date \_\_\_\_\_

**ARTWORK Proof for Approval**

- Although this proof has been checked in compliance with stringent quality assurance procedures, errors can occur.
- It is the customers responsibility to check this design for content, size, layout and colour separation prior to advising final approval to print.
- Please refer to Pantone reference book for accurate colour guides.
- Please check barcode for size, bwr and scannability.