

145mm

LBF® No Sting Barrier Film Spray – 30ml and 50ml

Product Description

LBF No Sting Barrier Film is dispersed as a non-stinging solvent, which dries rapidly forming a transparent protective film. LBF No Sting Barrier Film is indicated for the protection of intact skin that is at risk or likely to be compromised by corrosive bodily fluids (including urine, faeces, digestive enzymes), wound drainage, adhesives and friction.

Indications

LBF No Sting Barrier Film is a liquid intended for external use as a film-forming product. When applied to the skin, it forms a protective barrier between the skin and corrosive bodily fluids, adhesive products and friction. It is intended as a primary barrier against irritation from bodily fluids. The product may be used on adults, children and infants over 12 months of age.

Contraindications

LBF No Sting Barrier Film is not to be used as the only covering in situations that require dressing protection from bacterial contamination / penetration, e.g. intravenous therapy catheter sites and full or partial thickness wounds. It is not to be used on infected skin or in cases where there is a known sensitivity to any of the ingredients.

Warning

LBF No Sting Barrier Film, in liquid form, is flammable; use in well ventilated area. Avoid using around flames and sources of ignition. Keep out of reach of children. LBF No Sting Barrier Film is for external use only. If eye contact / ingestion occur seek immediate medical attention. Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina etc.). Not advised for use near tracheostomies.

Precautions

1. Should redness or other signs of irritation appear, or if redness or signs of irritation persist, consult a medical professional.
2. Use of other barrier products, ointments, creams or lotions may significantly reduce the effectiveness of LBF No Sting Barrier Film.
3. Do not use if the packaging is damaged or opened.

Instructions For Use

1. Skin should be clean and dry prior to application of LBF No Sting Barrier Film.
2. Hold the spray nozzle 10-15cm (4-6 inches) from the skin and apply a smooth, uniform coating of film over the entire area of concern, while moving the spray in a sweeping motion. Do not spray directly into wound.
3. If an area of skin is missed, reapply to that area only after the first application of LBF No Sting Barrier Film has dried (approximately 30 seconds).
4. If LBF No Sting Barrier Film is applied to an area with skin folds or other skin-to-skin contact, make sure that skin-contact areas are separated to allow them to dry before returning to normal position.

1649/1112/1

205mm

5. When used under adhesives tapes, dressings or devices:
 - a). Allow LBF No Sting Barrier Film to dry before covering with dressings or adhesive products.
 - b). Reapplication of LBF No Sting Barrier Film is necessary each time dressing and / or adhesive products are changed; the barrier is removed by the adhesive.
6. When used as a protectant against bodily fluids, faeces or urine and no adhesive products are applied to the skin:
 - a). Reapplication of LBF No Sting Barrier Film is recommended every 24-48 hours, depending on the frequency of cleaning.
 - b). In extreme cases (e.g. constant diarrheal stooling) with very frequent cleansing, more frequent applications may be necessary (i.e. every 12-24 hours).
7. If desired, the film can be removed by using most medical adhesive removers as directed. Clean and dry the involved area and reapply LBF No Sting Barrier Film.
8. Replace cap over spray nozzle when not in use to prevent accidental dispensing.

Ingredients

Siloxane copolymers.

Storage and Shelf Life

Store away from direct sunlight, ambient temperature and humidity. For shelf life, refer to the expiry date on each can.



Ordering Information

Product Code	Description	No. per box
3826	LBF No Sting Barrier Film Spray, 50ml	1
3827	LBF No Sting Barrier Film Spray, 30ml	1

CliniMed Ltd., Cavell House,
Knaves Beech Way, Loudwater,
High Wycombe, Bucks HP10 9QY
Tel 01628 850100 Fax 01628 527312
www.clinimed.co.uk

CliniMed® and LBF® are registered trademarks of CliniMed (Holdings) Ltd.



0 10 20 30 40 50 60 70 80 90 100

Black	116	4B5
293	outlet	
Ink Count: 4		

Client	Flexible Medical	Date	6-December-12
Job Number	66106	Step Size	
Order Number		Operator	sarah.r
Job Title	LBF Leaflet		

approved _____	date _____
approved subject to _____	date _____

ARTWORK Proof for Approval

- Although this proof has been checked in compliance with stringent quality assurance procedures, errors can occur.
- It is the customers responsibility to check this design for content, size, layout and colour separation prior to advising final approval to print.
- Please refer to Pantone reference book for accurate colour guides.
- Please check barcode for size, bwr and scannability.