

Safe and Smart



Large loop for opening or hanging.

Adhesive area on back for hanging up the package.

Sterile water. A press on the water pocket makes the water run down and activate the catheter's surface layer.

Insertion guide. A firm grip without needing to touch the catheter with your fingers.

Hydrophilic catheter with Urotonic[™] Surface Technology for minimal friction during insertion and withdrawal.



Foldable and discreet.

Smooth catheter eyes.

Intended use

The LoFric[®] Primo[™] single use urinary catheter is intended for intermittent urinary catheterisation.

Sterilisation and shelf life

All catheters are sterilised by irradiation. Information about the recommended best before date is printed on the package.

Package

30 catheters per carton and 4 cartons per shipping box.

Material composition

Hydrophilic surface coating: Polyvinyl pyrrolidone (PVP) and sodium chloride (NaCl).

Catheter: Medical grade polyvinyl chloride, PVC.

Water pocket: Laminate foil of polyethylene terephthalate (PET), polyethylene (PE) and aluminium, sterile water.


Item package: Polyethylene (PE).

Product Range


Reference	Type	Length	Size
960 08 00	LoFric [®] Primo [™] Male	40 cm	CH 08
960 10 00	LoFric [®] Primo [™] Male	40 cm	CH 10
960 12 00	LoFric [®] Primo [™] Male	40 cm	CH 12
960 14 00	LoFric [®] Primo [™] Male	40 cm	CH 14
960 16 00	LoFric [®] Primo [™] Male	40 cm	CH 16
960 18 00	LoFric [®] Primo [™] Male	40 cm	CH 18
962 06 00	LoFric [®] Primo [™] Paediatric	20 cm	CH 06
962 08 00	LoFric [®] Primo [™] Paediatric	20 cm	CH 08
962 10 00	LoFric [®] Primo [™] Paediatric	20 cm	CH 10
964 08 00	LoFric [®] Primo [™] Female	20 cm	CH 08
964 10 00	LoFric [®] Primo [™] Female	20 cm	CH 10
964 12 00	LoFric [®] Primo [™] Female	20 cm	CH 12
964 14 00	LoFric [®] Primo [™] Female	20 cm	CH 14
964 16 00	LoFric [®] Primo [™] Female	20 cm	CH 16
964 18 00	LoFric [®] Primo [™] Female	20 cm	CH 18
968 08 00	LoFric [®] Primo [™] Female	15 cm	CH 08
968 10 00	LoFric [®] Primo [™] Female	15 cm	CH 10
968 12 00	LoFric [®] Primo [™] Female	15 cm	CH 12
968 14 00	LoFric [®] Primo [™] Female	15 cm	CH 14
966 10 00	LoFric [®] Primo [™] Tiemann	40 cm	CH 10
966 12 00	LoFric [®] Primo [™] Tiemann	40 cm	CH 12
966 14 00	LoFric [®] Primo [™] Tiemann	40 cm	CH 14
966 16 00	LoFric [®] Primo [™] Tiemann	40 cm	CH 16
966 18 00	LoFric [®] Primo [™] Tiemann	40 cm	CH 18

Instructions


Opening 1 (Sterile opening)

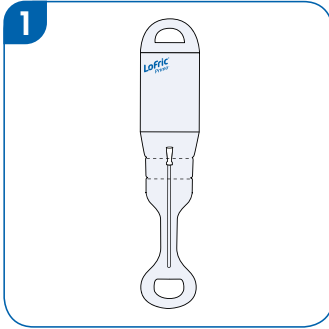


1. Unfold the package. Hold the product upright.

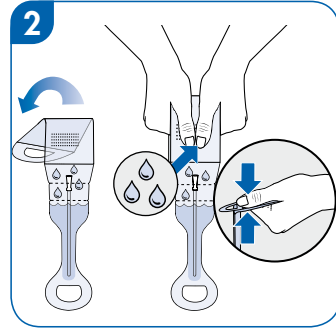

2. Fold the water pocket. Squeeze the water pocket and let the water run down to the catheter.

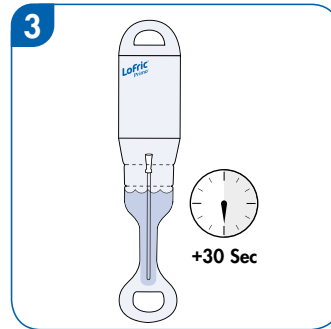


3. Hang LoFric Primo using the upper loops or the adhesive area. Let it soak for 30 seconds while you prepare for insertion.

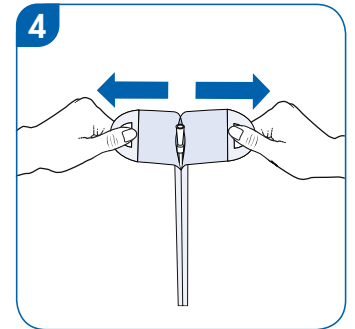


4. Peel open packaging. Grip the package at the loops above the water pocket with both hands. Peel until the connector is exposed. Catheterise.


Opening 2 (Using the insertion guide)

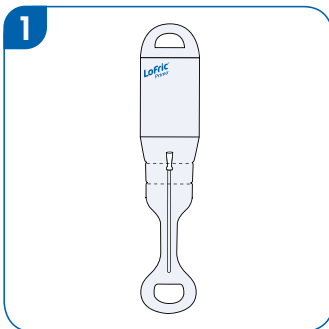


1. Unfold the package. Hold the product upright.

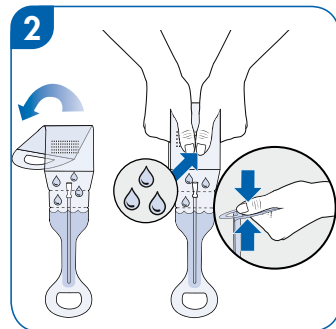


2. Fold the water pocket. Squeeze the water pocket and let the water run down to the catheter.

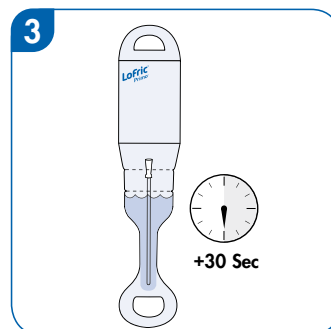

3. Hang LoFric Primo using the upper loops or the adhesive area. Let it soak for 30 seconds while you prepare for insertion.

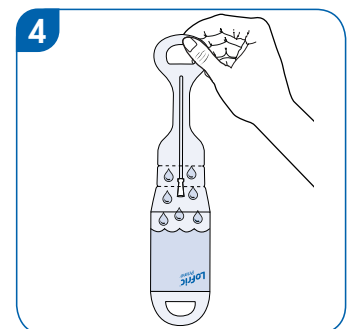


4. Turn the package upside down so the water flows back into the pocket.

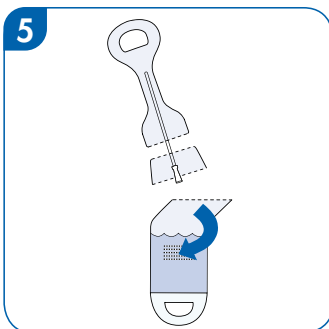

5. Remove the water pocket by tearing at indentation "A".

Hint! Tear half-way first, turn the package and tear the other half. Tear at indentation "B", in the same manner as for indentation "A".

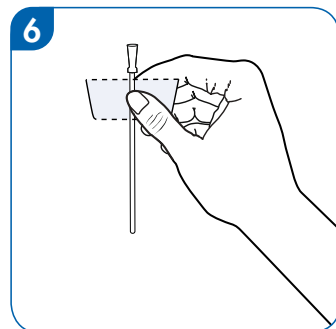

6. Use the remaining packaging part as a handling guide. This part will give you a firm and hygienic grip, allowing you to insert the catheter without touching it.