

Lyof foam[®] Max

A HIGHLY ABSORBENT POLYURETHANE FOAM DRESSING

Lyof foam Max is a highly absorbent dressing. It consists of a polyurethane foam and a waterproof backing film with high permeability to allow for an effective exudate management of moderate to highly exuding wounds.

POLYURETHANE BACKING FILM

- Moisture vapour permeable
- Waterproof
- Passed viral penetration test
- Passed wet bacterial resistance test



HYDROPHILIC POLYURETHANE FOAM

- High absorbency and fluid retention capacity
- Works under compression
- Soft, conformable and easy to apply and remove

AREAS OF USE


Lyof foam® Max is designed for the management of moderate to heavily exuding wounds such as leg ulcers and pressure sores.

Through an excellent exudate management, Lyof foam Max creates a moist wound environment for optimal wound healing to take place.

KEY BENEFITS OF LYOFOAM MAX

- High absorbency and fluid retention capacity¹
- Reduced risk of maceration, less potential for leakage and longer wear-time¹
- Waterproof backing film acts as a barrier to bacteria and virus penetration
- Works under compression
- Easy to apply and remove
- Soft and conformable and provides comfort to patients
- Non-sensitising
- Wide range of sizes available for different wound types


HOW TO USE LYOFOAM MAX

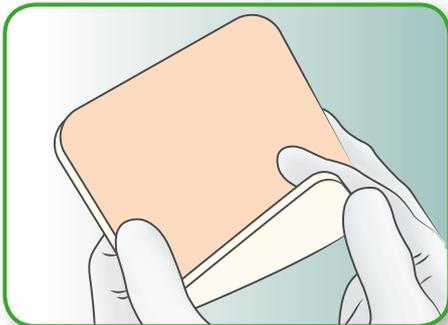

1. Clean the wound area and select a suitable size of Lyof foam Max.



2. Place the foam side directly onto the wound surface, overlapping the wound margin by about 2cm.

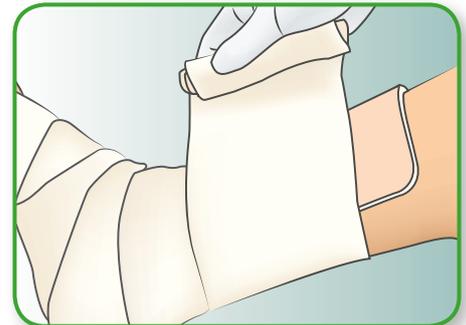

3. Secure Lyof foam Max with an appropriate bandage such as Tubifast, Tubigrip, System 4 or a fixation tape such as Mefix.

FREQUENCY OF CHANGE

Lyof foam Max may be left in place for several days depending on the nature and condition of the wound and the level of exudate.



LYOFOAM MAX ORDERING INFORMATION

Art. No	Size cm	Pieces Per Inner	PIP Code	NHS Code
603200	7.5 x 8.5	10	367-8778	ELA571
603201	10 x 10	10	367-8786	ELA572
603203	10 x 20	10	367-8794	ELA573
603204	15 x 15	10	367-8802	ELA574
603205	15 x 20	10	367-8810	ELA575
603206	20 x 20	10	367-8828	ELA576



References:

1. SMTL reports - Report No. 11/3674/1 - Report No. 11/3571/2 - Report No. 11/3571/3 - Report No. 11/3571/6 - Report No. 11/3571/7

Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, www.molnlycke.co.uk

The Mölnlycke Health Care name and logo, Lyof foam® Max are registered trademarks of Mölnlycke Health Care AB. ©2012 Mölnlycke Health Care. All rights reserved.

UK257761202