



SELF-ADHESIVE FABRIC

Mefix is the ideal fixation for dressings, swabs, catheters, and tubes. It provides secure fixation and the adhesive is gentle to the skin.

Pre-measured protection paper for convenient measurement and cutting



Skin-friendly, waterbased, solvent-free polyacrylate adhesive for secure and gentle fixation

Soft, elastic non-absorbent non-woven for patient comfort

HOW MEFIX WORKS

The porous structure of the nonwoven fabric ensures air and water vapour permeability. The fabric is flexible, conforms well to body contours, and allows body movements when in place. The fabric is non-absorbent and provides a barrier for blood and fluid strike-through.


The water-based polyacrylate adhesive has been designed to be gentle to the skin and at the same time provide secure fixation.

The easy-to-remove protection paper is pre-measured in 10 cm intervals, and has a perforated split for easy removal and application. Mefix can easily be cut to the desired shape and size.

AREAS OF USE

Mefix can be used in a wide variety of situations where fixation is required, including fixation of dressings, swabs, catheters, and tubes.


HOW TO USE MEFIX- FIXATION USING THE S-CURVE

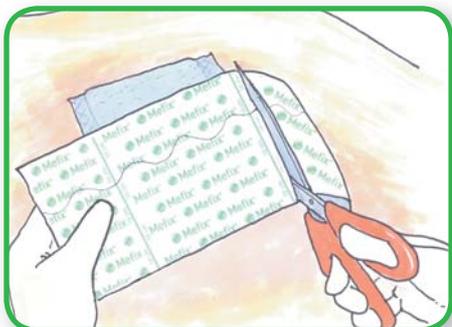

1. Cut to appropriate size.



2. Separate the two parts of the protection paper by lightly stretching and bending the Mefix. Remove the narrow part.

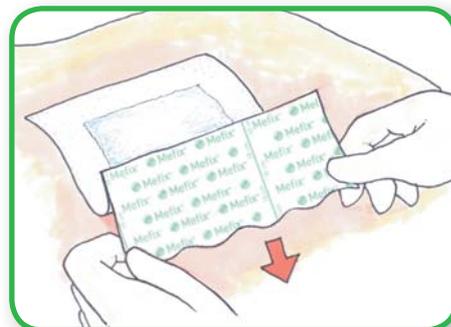

3. Apply the part with the exposed adhesive to the skin. Remove the remaining protection paper. Firmly smooth Mefix to obtain proper adhesion. Do not stretch Mefix when applying.

BENEFITS OF MEFIX

- Skin-friendly, water-based, solvent-free adhesive
- Pre-measured protection paper allows for convenient measurement and cutting
- Adapts well to body contours
- Air permeable, prevents maceration and gives better comfort
- Can be sterilized by steam and EtO



PRECAUTIONS

Care should be taken to ensure that Mefix is not applied under tension, to prevent shearing forces causing damage to the skin. This is particularly important when applied over joints.

MEFIX ORDERING INFORMATION

Art. No	Size cm x m	Pieces Per Inner	PIP Code	NHS Code
310250	2.5cm x 10m	1	N/A	EHR000
310500	5cm x 10m	1	N/A	EHR001
311000	10cm x 10m	1	N/A	EHR002
311500	15cm x 10m	1	N/A	EHR003
312000	20cm x 10m	1	N/A	EHR004
310276	2.5cm x 5m	1	268-4033	N/A
310576	5cm x 5m	1	268-4793	N/A
311076	10cm x 5m	1	268-4801	N/A
311576	15cm x 5m	1	268-4819	N/A
312076	20cm x 5m	1	268-4827	N/A
313076	30cm x 5m	1	268-4835	N/A



**MÖLNLYCKE
HEALTH CARE**

Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, www.molnlycke.co.uk

The Mölnlycke Health Care name and logo, Mefix® are registered trademarks of Mölnlycke Health Care AB. ©2009 Mölnlycke Health Care. All rights reserved.

UK121550611