


# ● Mepilex® Border Lite

## THE THIN SELF ADHERENT DRESSING WITH SAFETAC® TECHNOLOGY

- Atraumatic to the wound and surrounding skin on removal
- The all-in-one solution

Viral & bacterial proof film backing

- Showerproof
- Breathable

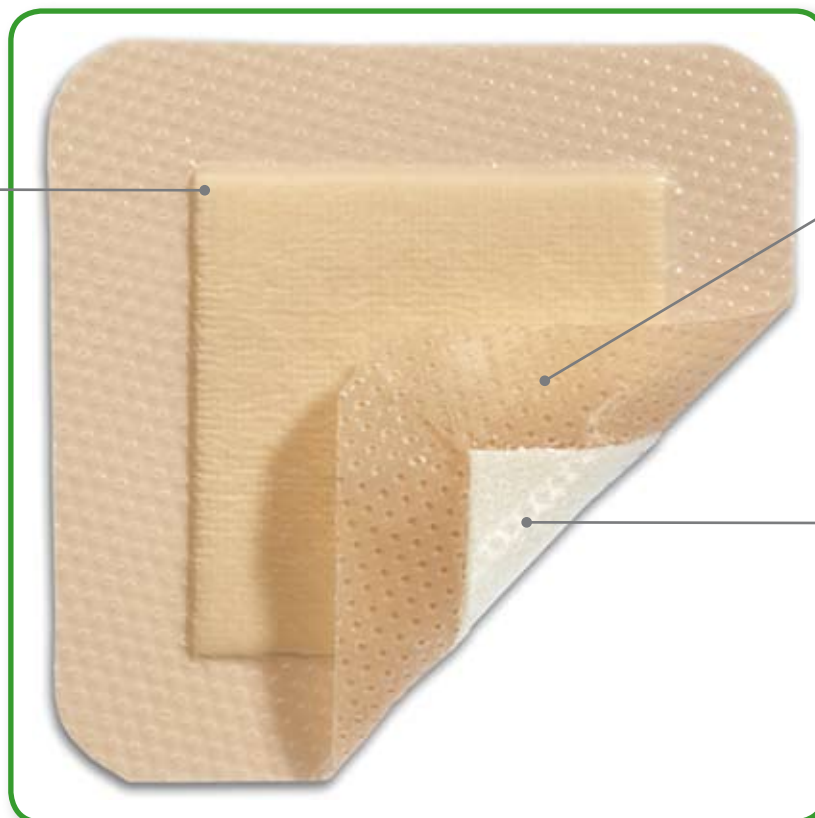

Layer with Safetac Technology

- Secure fixation
- Atraumatic to the wound bed and surrounding skin on removal
- Reduces risk of maceration

Multi-layered thin foam pad

- Hydrophilic polyurethane foam to quickly absorb exudate
- Non-woven spreading layer to optimise the absorption capacity and prevent leakage

# BENEFITS OF MEPILEX BORDER LITE

Mepilex Border Lite minimises trauma to the wound and pain to the patient during dressing changes. The Safetac technology prevents Mepilex Border Lite from sticking to the wound bed yet adheres gently to the surrounding skin.


Mepilex Border Lite is ideal for those locations where a thin self-adherent dressing is required. It is a very conformable and gentle alternative to other thin adhesive bordered dressings.

# AREAS OF USE


A wide range of non to low exuding wounds such as:

- Diabetic foot ulcers
- Wounds with comprised/fragile surrounding skin
- Painful wounds
- Acute wounds
- Surgical wounds

# HOW TO USE MEPILEX BORDER LITE

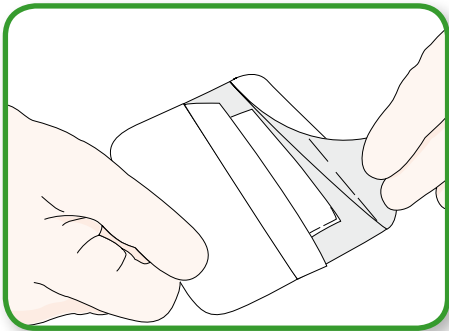

1. Clean the wound area. Remove the release film.

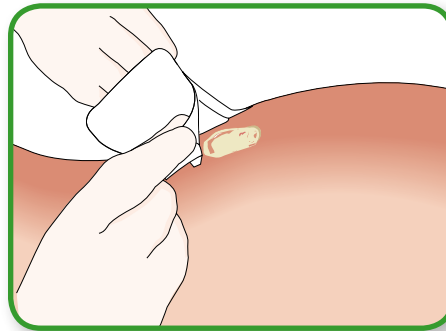

2. Apply the adherent side to the wound. Do not stretch.

# NOTE

In case of clinical signs of infection the use of Mepilex Border Lite may be continued if proper infection treatment is initiated.



## MEPILEX BORDER LITE ORDERING INFORMATION

Art. No	Size cm	Pieces Per Inner	PIP Code	NHS Code
281000	4 x 5	10	317-7011	ELA272
281100	5 x 12,5	5	317-7029	ELA273
281200	7.5 x 7.5	5	317-7037	ELA269
281300	10 x 10	5	317-7045	ELA270
281500	15 x 15	5	317-7060	ELA271

