

Mepilex[®] Border

Safetac[®]
TECHNOLOGY

New improved Mepilex[®] Border with Safetac[®] technology

For years, Mepilex Border has proven to be an effective and reliable self-adherent dressing for low to highly exuding wounds. It features the unique and patented Safetac technology that minimises trauma and pain at dressing changes.


The improved Mepilex Border has even better fluid handling and retention so that you don't have to worry about leakage and maceration. Improved Mepilex Border gives you the following advantages:

Less friction against clothes and bed linens for more secure fixations

More transparent border makes the dressing more discrete

Viral proof backing film, protects from outer contamination.
[Has passed viral penetration test ASTM F1671]

Improved fluid handling, reducing the risk for leakage and maceration

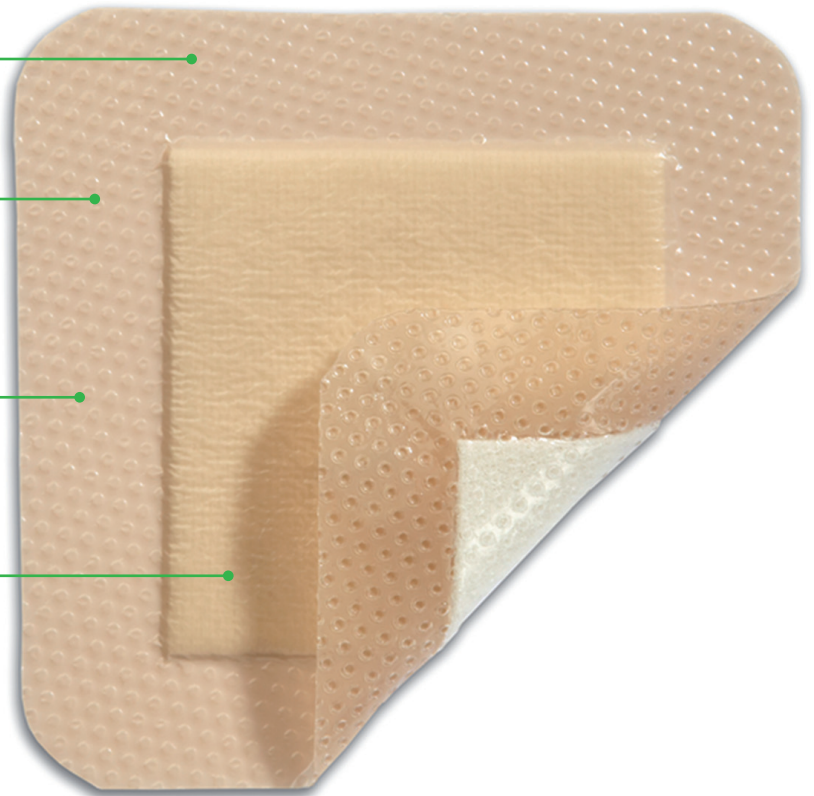


Safetac[®] technology


Safetac[®] technology. Less pain and less trauma.

Safetac is a patented adhesive technology that minimises pain to patients and trauma to wounds. Safetac technology is available exclusively on Mölnlycke Health Care dressings, including Mepilex[®] wound care dressings, Mepitel[®], Mepiform[®] and Mepitac[®].

For more information visit www.safetac.com

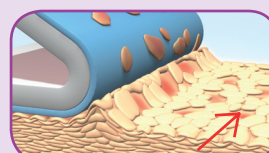


Skin stripping occurs with traditional adhesive

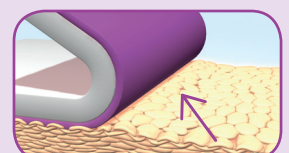


No skin stripping occurs with Safetac technology

How to use Mepilex Border

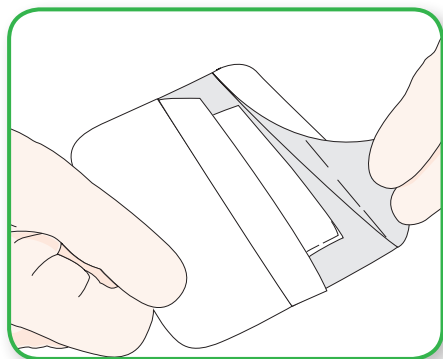

1. Clean the wound area. Remove the release film.

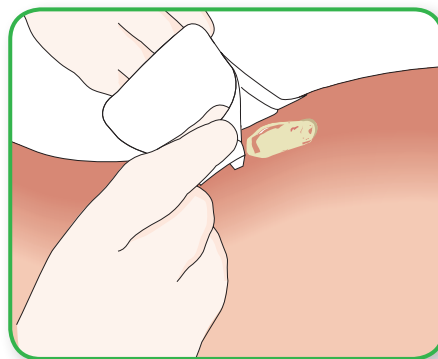

2. Apply the adherent side to the wound. Do not stretch.

How Mepilex Border works

The self-adherent Mepilex Border absorbs exudate effectively and minimises the risk of maceration. The dressing maintains a moist wound environment for optimal wound healing. The Safetac properties allows the dressing to be changed without causing additional pain to the patient, or trauma to the wound or surrounding skin.

Benefits of Mepilex Border

- Self-adherent
- Does not adhere to moist wound beds but gently to dry tissue
- Minimises trauma and pain during dressing changes
- Can remain in place for up to seven days depending on the condition of the wound
- Maintains a moist wound environment
- Minimises the risk of maceration
- Promotes patient comfort during wear
- Can be used under compression bandages

● Mepilex® Border



● Mepilex® Border Sacrum



Areas of use

Mepilex Border is designed for a wide range of exuding wounds such as:

- Pressure ulcers
- Leg and foot ulcers
- Painful Wounds
- Wounds with compromised/fragile surrounding

Note

In case of clinical signs of infection the use of Mepilex Border may be continued if proper infection treatment is initiated.

Mepilex Border Ordering Information

Art. no	Size cm	Pieces per inner	PIP Code	NHS Code
295260	7 x 7.5	5	339-0986	ELA381
295360	10 x 12.5	5	339-0978	ELA380
295800	10 x 20	5	327-1590	ELA186
295900	10 x 30	5	327-1608	ELA187
295460	15 x 17.5	5	339-0952	ELA379
295660	17 x 20	5	339-0945	ELA078

Sterile packed

Mepilex Border Sacrum Ordering Information

Art. no	Size cm	Pieces per inner	PIP Code	NHS Code
282500	15 x 15	5	366-7532	ELA577
282000	18 x 18	5	327-3950	ELA300
282400	23 x 23	5	327-1616	ELA173

References:

Clinical reference available on request.

0800 7311 876 • info.uk@molnlycke.com • www.molnlycke.co.uk
Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable,
Bedfordshire, LU5 5UL, UK

The Mölnlycke Health Care name and logo, Safetac®, Mepilex® Border, Mepilex®, Mepitel®, Mepiform® and Mepitac® are registered trademarks of Mölnlycke Health Care AB. ©2012 Mölnlycke Health Care. All rights reserved.

UK166040611