


Mepilex[®] Lite

THE THIN DRESSING WITH SAFETAC[®] TECHNOLOGY


- Atraumatic to the wound and surrounding skin on removal
- Conforms well to body contours

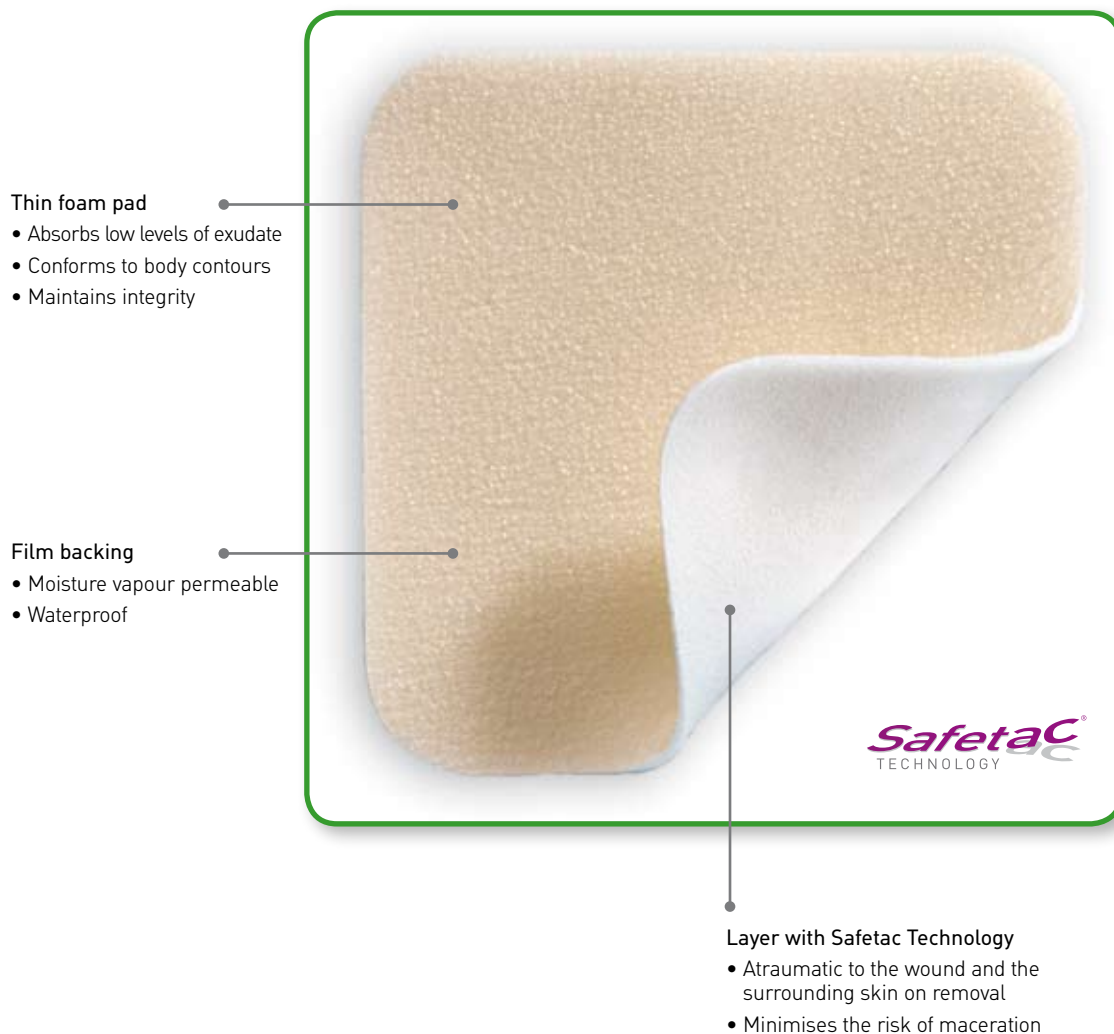

HOW MEPILEX LITE WORKS

Mepilex Lite is a highly conformable dressing that absorbs low levels of exudate and maintains a moist wound environment. The Safetac layer seals around the wound edges, preventing the exudate leaking onto the surrounding skin, thus minimising the risk of maceration. The Safetac layer ensures atraumatic dressing changes. Mepilex Lite can be cut to suit various wound shapes and locations.

HOW TO USE MEPILEX LITE

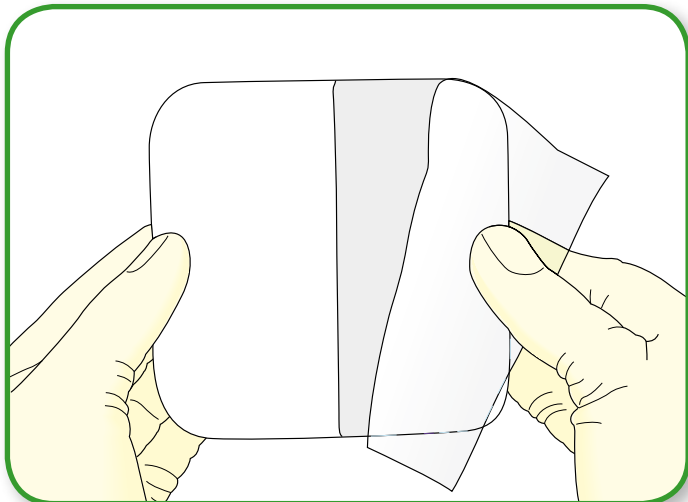

1. Cleanse the wound in accordance with normal procedures. Dry the surrounding skin thoroughly. Use release films for handling during application.



2. Apply the adherent side to the wound. Do not stretch. Mepilex Lite should overlap dry surrounding skin by at least 2cm. If required Mepilex Lite can be cut.

BENEFITS OF MEPILEX LITE

- Atraumatic to the wound and surrounding skin on removal
- Minimises pain to the patient on removal
- Minimises the risk of maceration
- Conforms well to body contours
- Thin and comfortable to wear
- Easy to handle
- Maintains a moist wound environment
- Can be cut
- Can be left in place during radiotherapy using commonly applied energy ranges and absorbed doses



AREAS OF USE

Mepilex Lite is designed for the management of a wide range of non to low exuding wounds, such as leg and foot ulcers, pressure ulcers, partial thickness burns, radiation skin reactions and Epidermolysis Bullosa (EB). Mepilex Lite can also be used as protection of compromised and/or fragile skin. Mepilex Lite can be used under compression bandaging. Mepilex Lite can be used in combination with gels.

MEPILEX LITE ORDERING INFORMATION

Art. No	Size cm	Pieces Per Inner	PIP Code	NHS Code
284000	6 X 8.5	5	303-5565	ELA182
284100	10 X 10	5	303-5573	ELA184
284300	15 X 15	5	303-5581	ELA185
284500	20 X 50	2	311-6787	ELA234



Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, www.molnlycke.co.uk

The Mölnlycke Health Care name and logo, Safetac® and Mepilex® Lite are registered trademarks of Mölnlycke Health Care AB. ©2009 Mölnlycke Health Care. All rights reserved.

UK165030611