



ABSORBENT DRESSING WITH SAFETAC[®] TECHNOLOGY

- Atraumatic to the wound and surrounding skin on removal
- Absorbs effectively
- Can be used under compression
- Minimises the risk of maceration



AREAS OF USE

Mepilex is designed for a wide range of exuding wounds such as leg and foot ulcers, pressure ulcers and traumatic wounds, e.g. skin tears and secondary healing wounds.

NOTE

In case of clinical signs of infection the use of Mepilex may be continued if proper infection treatment is initiated.

HOW TO USE MEPILEX



1. Clean the wound area. Remove the release film.



2. Mepilex should overlap the wound bed by at least 2 cm onto the surrounding skin. Apply the adherent side to the wound. Do not stretch.



3. When necessary, secure Mepilex with a bandage or other fixation.

BENEFITS OF MEPILEX

- Atraumatic to the wound and surrounding skin on removal
- Can remain in place for several days depending on the condition of the wound
- Minimises the risk of maceration
- Promotes patient comfort during wear
- Can be used under compression bandages
- Can be cut to size

MEPILEX ORDERING INFORMATION

Art. no	Size cm	Pieces per inner	PIP Code	NHS Code
294160	10 x 11	5	339-1109	ELA378
294260	11 x 20	5	339-1091	ELA085
294360	15 x 16	5	339-1083	ELA333
294460	20 x 21	5	339-1034	ELA334
294500	20 x 50	2	342-9537	ELA383



Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, www.molnlycke.co.uk
The Mölnlycke Health Care name and logo, Safetac® and Mepilex® are registered trademarks of Mölnlycke Health Care AB. ©2009 Mölnlycke Health Care. All rights reserved.

UK164020611