




THE TAPE WITH SAFETAC® TECHNOLOGY

- Atraumatic to the skin on removal
- Ideal fixation for patients with fragile skin
- Conforms well to body contours

- Knitted fabric**
 - Conformable
 - Easy to cut
- Film backing**
 - Breathable
 - Showerproof

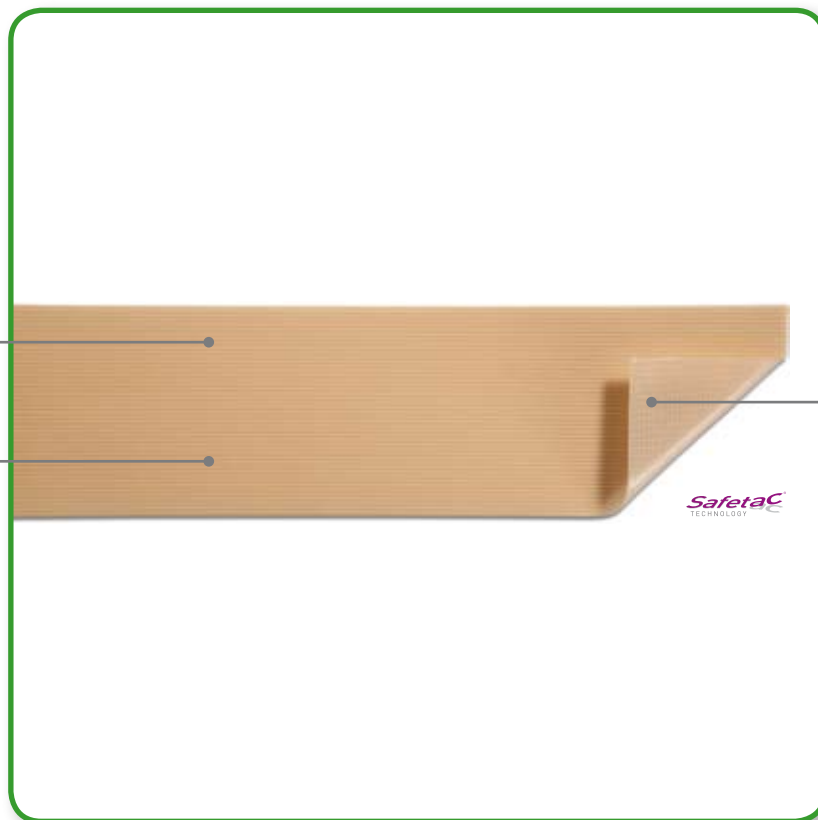


- Layer with SafetaC Technology**
 - Atraumatic to the skin on removal

HOW MEPITAC WORKS


Mepitac is highly conformable and easy to apply and remove. The Safetac layer assures secure fixation and atraumatic removal from skin and bodily hair. If inspection or repositioning is necessary, Mepitac can be lifted and re-applied.

Mepitac is an ideal fixation solution for patients with fragile and sensitive skin. Mepitac is suitable for patients requiring repeated application and removal of tape over the same area e.g. neonates and dialysis patients.


HOW TO USE MEPITAC

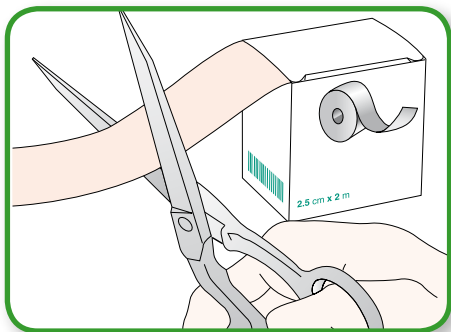

1. Make sure the skin is clean and dry and free from any ointment etc. Cut off the desired length of tape using scissors.

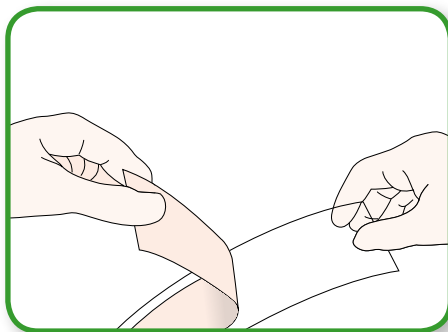

2. Remove the protective release film .

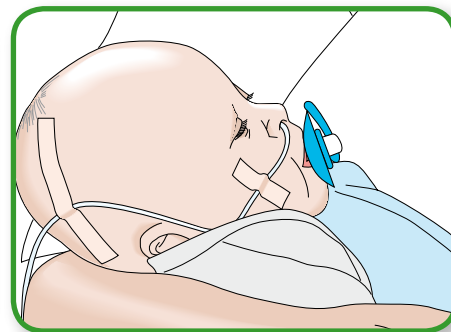

3. Apply Mepitac without stretching. If inspection or repositioning is necessary, Mepitac can be lifted and re-applied.

BENEFITS OF MEPITAC

- Atraumatic to the skin on removal
- Conforms well to body contours
- Leaves no residues
- Showerproof
- Breathable



NOTE

The adhesion of Mepitac may decrease when used in humid condition such as incubators or on patients who have a fever or perspire a lot. Mepitac is not suitable for fixation of arterial catheters and arterial cannulae.

MEPITAC ORDERING INFORMATION

Art. No	Size cm	Pieces Per Inner	Quantity Per Case	PIP Code	NHS Code
298300-00	2 x 300	1	12	308-9125	EHR041
298400-00	4 x 150	1	12	308-9133	EHR042



Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, www.molnlycke.co.uk

The Mölnlycke Health Care name and logo, Safetac® and Mepitac® are registered trademarks of Mölnlycke Health Care AB. ©2009 Mölnlycke Health Care. All rights reserved.

UK170450611