


# Mepore® Film

## BREATHABLE, TRANSPARENT SELF-ADHESIVE FILM DRESSING

Mepore Film is a transparent self-adhesive film dressing ideal as a secondary fixation dressing and as a primary dressing for a wide range of clean wounds. Mepore Film's application system is designed for ease-of-use.

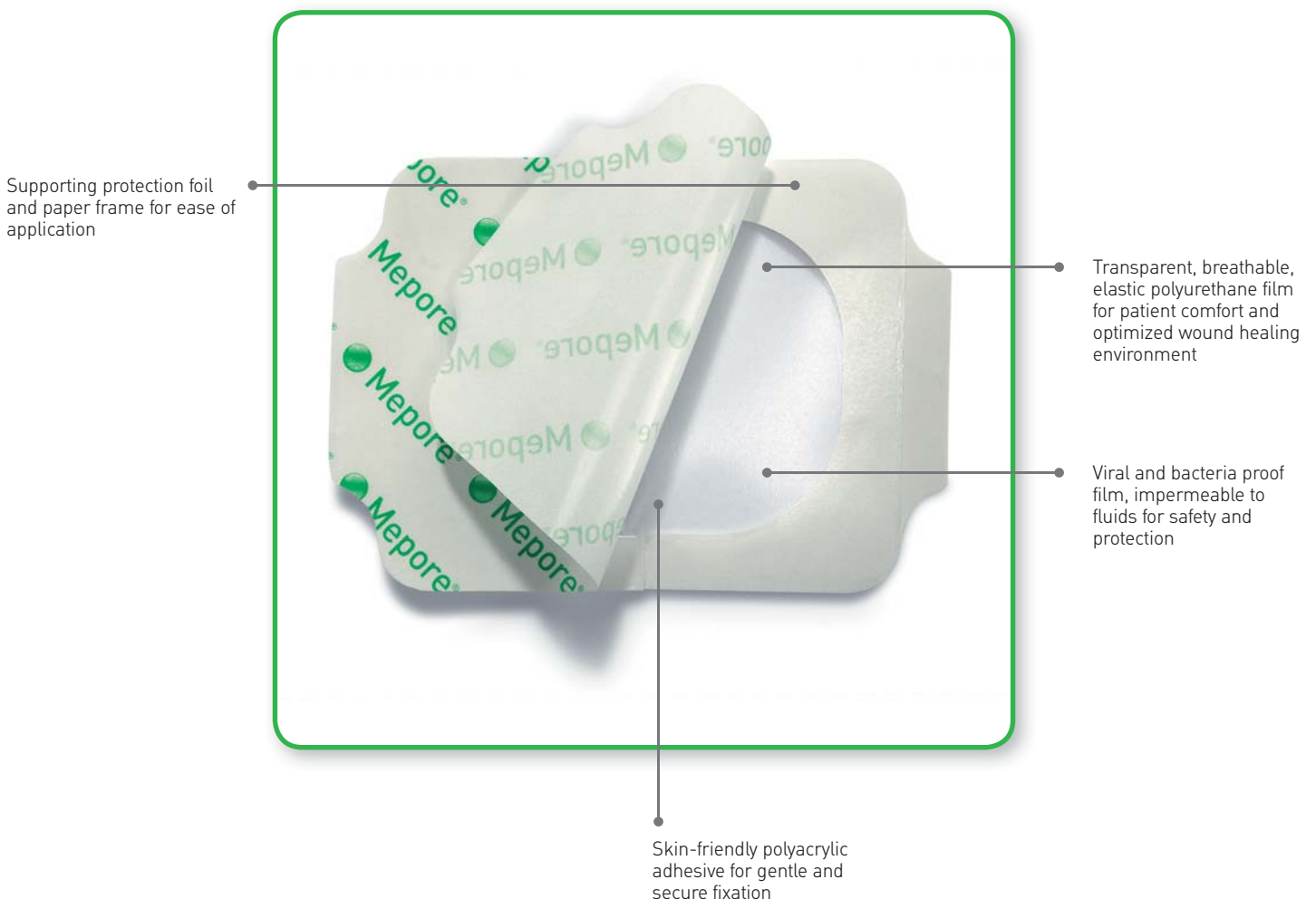

## HOW MEPORE FILM WORKS

Mepore Film provides a flexible, transparent covering to protect the wound, whilst conforming to surface irregularities and body contours. The dressing provides a barrier to fluid strike-through and viral and bacterial contamination.

Mepore Film maintains a moist environment in the wound. Due to its high vapour permeability, Mepore Film allows excess moisture to pass away from the wound and surrounding skin.

## AREAS OF USE

Mepore Film can be used as a conformable fixation dressing to cover primary dressings, as e.g. hydrogels, alginates, and foam dressings.

Mepore Film is also designed for a wide range of clean wounds in the granulation phase, such as:

- Superficial burns
- IV sites
- Abrasions
- Lacerations
- Donor sites
- Superficial pressure ulcers
- Closed surgical wounds
- Prevention of skin breakdown

## HOW TO USE MEPORE FILM



1. For the size 10 x 25 cm and 15 x 20 cm only: Remove center cut out paper and discard.



2. Remove the protection foil (printed Mepore) to expose the adhesive.



3. Position the dressing and firmly smooth it onto the skin. Remove the paper frame. Remove the two white paper side tabs. Do not stretch the dressing when applying.

## BENEFITS OF MEPORE FILM

- High breathability
- Easy application, also when wearing gloves
- Limits the risk for damage of newly formed tissue
- Conforms easily with body contours
- Skin-friendly adhesive
- Secure fixation
- Maintains a moist wound environment
- Viral and bacteria proof, and provides a barrier against leakage
- Fluid impermeable

## PRECAUTIONS

Mepore Film should not be used on full-thickness wounds involving muscle, tendon, bone or on third degree burns. Mepore Film should not be applied on patients who are sensitive to acrylic adhesive.

### MEPORE FILM ORDERING INFORMATION

| Art. No | Size cm | Pieces Per Inner | PIP Code | NHS Code |
|---------|---------|------------------|----------|----------|
| 270600  | 6 x 7   | 100              | N/A      | ELW133   |
| 271500  | 10 x 12 | 70               | N/A      | ELW210   |
| 272500  | 10 x 25 | 20               | N/A      | ELW136   |
| 273000  | 15 x 20 | 10               | 254-8758 | ELW135   |
| 270670  | 6 x 7   | 10               | 254-8725 | N/A      |
| 271570  | 10 x 12 | 10               | 254-8733 | N/A      |
| 272570  | 10 x 25 | 10               | 254-8741 | N/A      |
| 273500  | 20 x 30 | 5                | N/A      | ELW199   |



Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, [www.molnlycke.co.uk](http://www.molnlycke.co.uk)

The Mölnlycke Health Care name and logo, Mepore® Film are registered trademarks of Mölnlycke Health Care AB. ©2009 Mölnlycke Health Care. All rights reserved.

UK115550611