


Mepore® IV

TRANSPARENT, BREATHABLE IV FILM DRESSING


Mepore IV is a transparent self-adhesive IV dressing ideal for fixation of intravascular devices, such as IV catheters and other intravascular catheters. Mepore IV's application system is designed for ease-of-use.

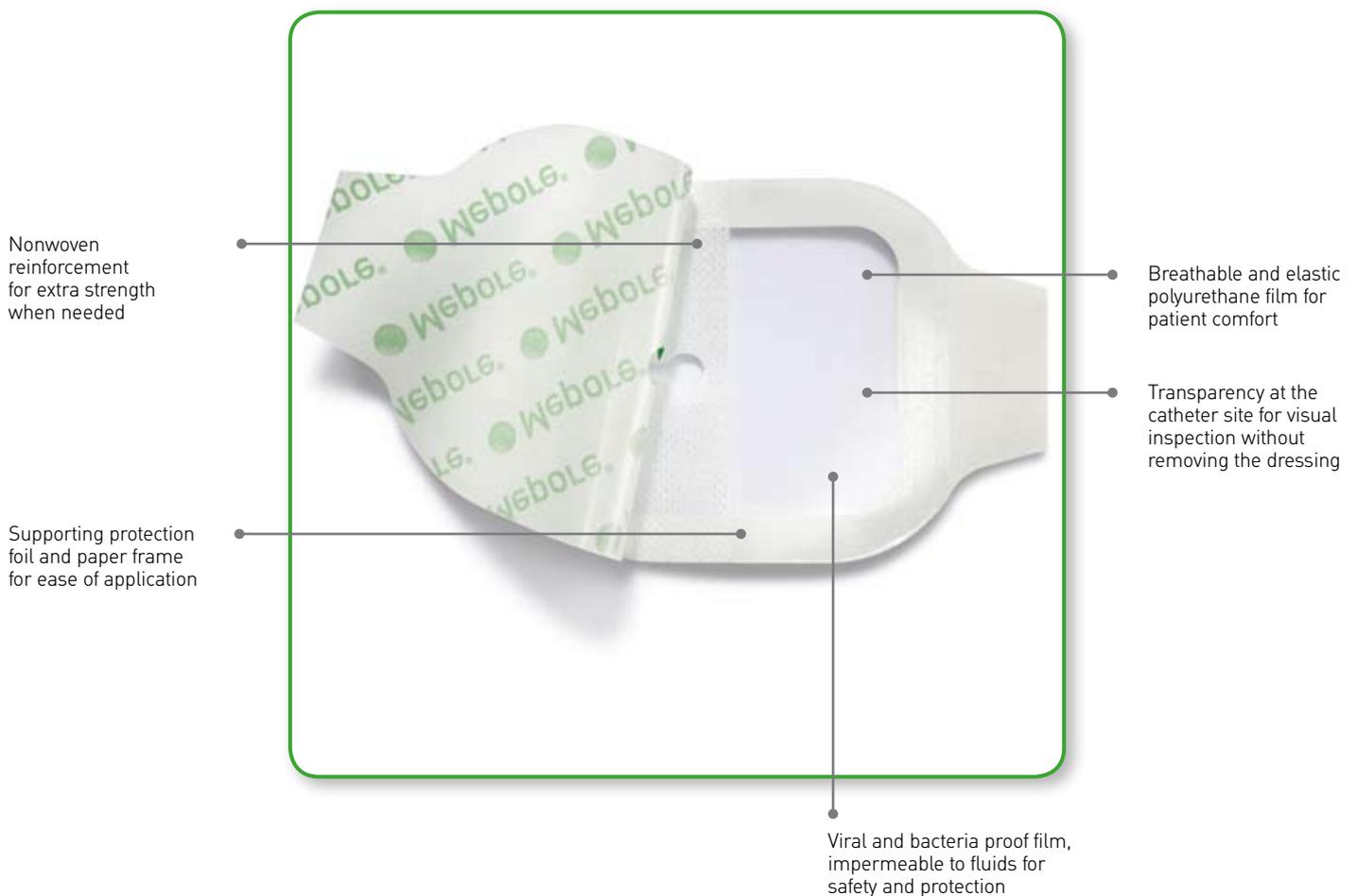


HOW MEPORE IV WORKS

Mepore IV provides a secure and flexible fixation of intravascular devices. The viral and bacteria proof transparent film gives a barrier to strike-through and outer contamination. Due to its high vapour permeability, Mepore IV allows excess moisture to pass away from the skin. The nonwoven reinforcement offers extra strength for reliable fixation. The polyacrylate adhesive has been designed to be gentle to the skin and at the same time provide secure fixation. The application system with the protection foil, paper frame, and generously sized side tab, ensures easy application.


HOW TO USE MEPORE IV

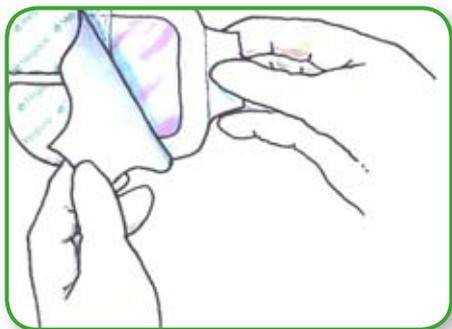

1. Open the package and remove the dressing. Hold the dressing by the side tab and remove the release paper.

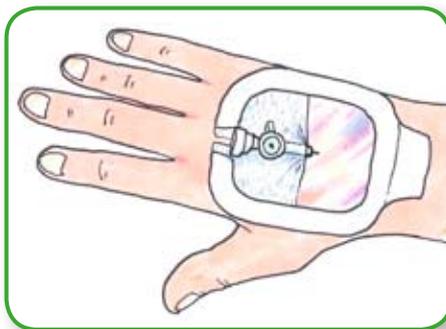

2. Make sure that the skin is clean and dry before application. Position the dressing over the catheter. Ensure that the insertion site is covered by the transparent film.

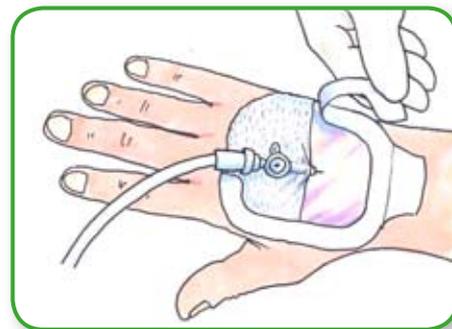

3. Remove the paper frame. Firmly smooth the dressing to obtain proper adhesion. Remove the side tab. Do not stretch the dressing when applying.

BENEFITS OF MEPORE IV

- Reliable and secure fixation
- Skin-friendly adhesive
- Nonwoven reinforcement for extra strength
- Breathable
- Transparent for visual inspections of the insertion site
- Conforms easily with body contours
- Bacteria and viral proof film
- Fluid impermeable
- Easy application, also when wearing gloves

MEPORE IV ORDERING INFORMATION

Art. No	Size cm	Pieces Per Inner	PIP Code	NHS Code
274000	5.5 x 5	100	326-9263	ELW224
274200	8 x 9	70	326-9271	ELW225
274400	10 x 11	70	326-9321	ELW226



Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, www.molnlycke.co.uk

The Mölnlycke Health Care name and logo, Mepore® IV are registered trademarks of Mölnlycke Health Care AB. ©2010 Mölnlycke Health Care. All rights reserved.

UK116550611