




SELF-ADHESIVE ABSORBENT DRESSING

Mepore is an air permeable, self-adhesive dressing ideal for low to moderately exuding wounds. Its skin-friendly, water-based, solvent-free polyacrylate adhesive provides a gentle and secure fixation.

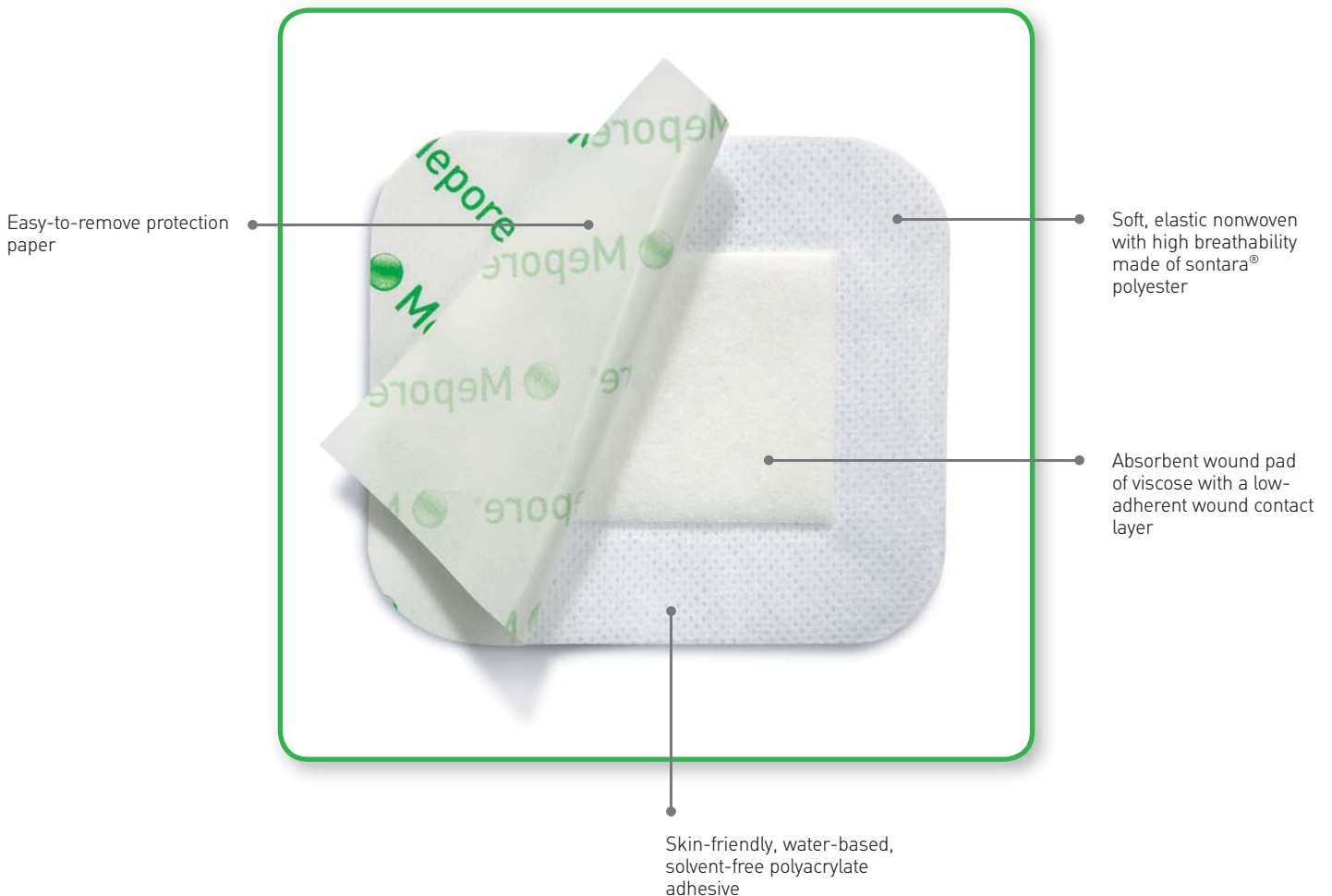

HOW MEPORE WORKS

The porous structure of the Sontara® nonwoven fabric ensures air and water vapour permeability. The fabric is flexible, conforms well to body contours, and allows body movements when in place. The fabric is non-absorbent and provides a barrier for blood and fluid strike-through. The polyacrylate adhesive has been designed to be gentle to the skin and at the same time provide secure fixation. The wound pad can absorb moderate amounts of wound fluids and the low-adherent wound contact layer reduces the

risk of adherence to the wound. The easy-to-remove protection paper permits sterile application, as well as simplifying dressing application.

AREAS OF USE

Mepore can be used in a wide variety of wounds which have low to moderate exudate levels. These include surgical wounds, minor burns, cuts, and abrasions.

HOW TO USE MEPORE



1. Open the package and remove the dressing.



2. Grip both overlapping release papers, exposing adhesive surface allowing initial fixation of the dressing.



3. Position the dressing on the skin without stretching and gently remove the rest of the release papers. Firmly smooth adhesive border to obtain proper adhesion. Do not stretch the dressing when applying.

FREQUENCY OF CHANGE

The frequency of dressing changes will be governed by the condition of the underlying wound. Upon removal grasp the edge and slowly peel the dressing from the skin in the direction of hair growth.

BENEFITS OF MEPORE

- Skin-friendly, water-based adhesive
- Low-adherent wound contact layer minimises risk of adherence to the wound
- Flexible fluid-repellent backing
- Easy-to-remove protection paper allows for convenient and aseptic application
- Air permeable, prevents maceration and gives better comfort
- Rounded corners for secure fixation (all sizes up to 9 x 15 cm)

PRECAUTIONS

Care should be taken to ensure that Mepore is not applied under tension, to prevent shearing forces causing damage to the skin. This is particularly important when the dressing is applied over joints.



MEPORE ORDERING INFORMATION

Art. No	Size cm	Pieces Per Inner	PIP Code	NHS Code
670700	7 x 8	55	284-2359	N/A
671500	10 x 11	40	284-2367	N/A
671600	11 x 15	40	284-2375	N/A
670800	6 x 7	60	N/A	EIJ500
670900	9 x 10	50	N/A	EIJ133
671000	9 x 15	50	N/A	EIJ000
671100	9 x 20	30	092-5305	EIJ137
671200	9 x 25	30	092-5347	EIJ139
671300	9 x 30	30	092-5354	EIJ141
671400	9 x 35	30	092-5396	EIJ143



**MÖLNLYCKE
HEALTH CARE**

Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, www.molnlycke.co.uk

The Mölnlycke Health Care name and logo, Mepore® are registered trademarks of Mölnlycke Health Care AB. ©2009 Mölnlycke Health Care. All rights reserved.

UK113550611