




ABSORBENT DRESSING WITH STRIKE-THROUGH EXUDATE BARRIER


Mesorb is a soft highly absorbent dressing that can retain large amounts of wound exudate. The fluid-repellent strike-through barrier helps prevent exudate soiling clothing and bed linen.

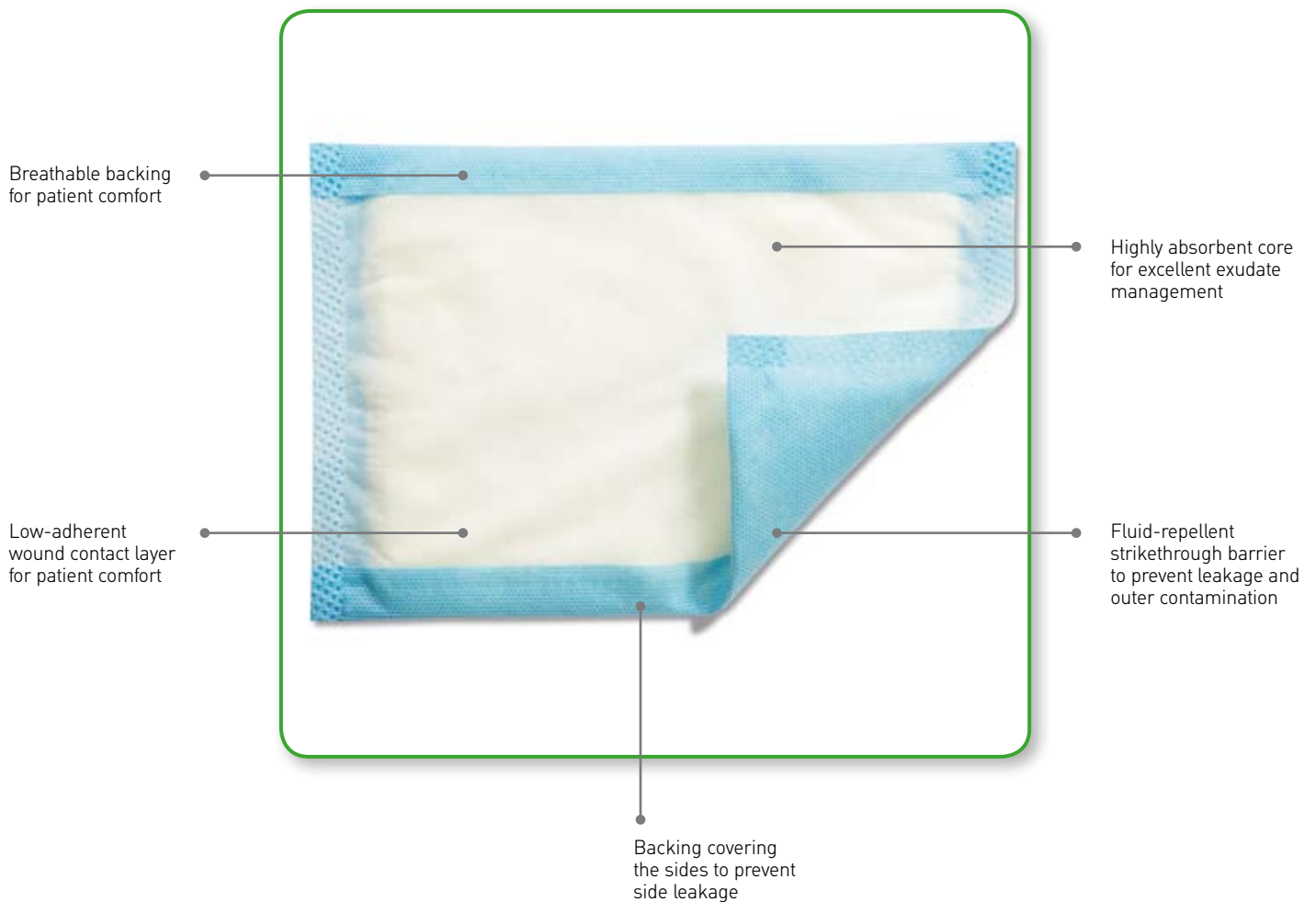

HOW MESORB WORKS

Wound exudate is absorbed and transferred into Mesorb by the wound contact layer and the diffusion layer. The highly absorbent core absorbs exudate, while simultaneously providing a good protective and ventilating wound cushion. The fluid-repellent backing prevents external contamination of the wound and protects clothes and bed linen from exudate strike-through. As the backing covers the sides of the dressing, it also prevents side leakage.


HOW TO USE MESORB

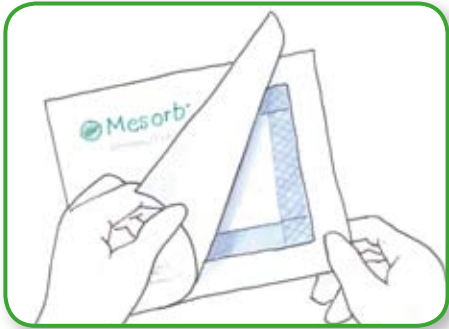

1. Open the package and remove the dressing.



2. Apply Mesorb on the wound area or on the chosen primary dressing with at least 1 cm overlap on the surrounding skin.

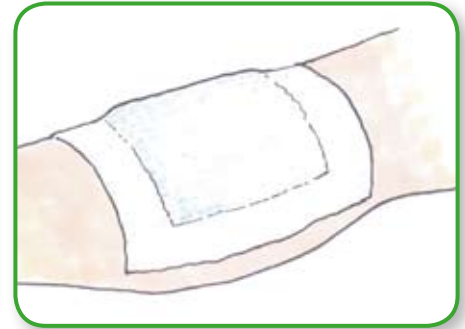

3. Fix in place with a suitable fixation device (e.g. Mefix®, Mepore® Film, or Tubifast™)

FREQUENCY OF CHANGE

Mesorb may be left in place for several days, depending on the wound and skin condition, or as indicated by a healthcare professional.

BENEFITS OF MESORB

- Highly absorbent
- Good protective cushioning
- Breathable
- Cost effective
- The unique strike-through exudate barrier reduces frequency of dressing changes and prevents soiling of clothing and bed linen



PRECAUTIONS

Mesorb should not be used on infected wounds without consulting a health care professional.

MESORB ORDERING INFORMATION

Art. No	Size cm	Pieces Per Inner	PIP Code	NHS Code
677080	10 x 10	10	255-3527	EJE044
677280	10 x 15	10	255-3535	EJE056
677480	10 x 20	10	255-3543	EJE058
677680	15 x 20	10	255-3550	EJE059
677780	20 x 25	10	255-3568	EJE060
677880	20 x 30	10	255-3576	EJE061



Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire LU5 5UL, UK. Tel. +44 (0) 870 6060766, www.molnlycke.co.uk

The Mölnlycke Health Care name and logo, Mesorb® are registered trademarks of Mölnlycke Health Care AB
© 2012 Mölnlycke Health Care. All rights reserved.

UK117550611