



Reference Guide for MicroClave[®] Clear Neutral Displacement Connector



Disinfect to Protect

- › When placing a new MicroClave Clear on a catheter, disinfect the catheter hub and prime the MicroClave Clear.
- › Before accessing a MicroClave Clear, always disinfect the injection site with the approved antiseptic per facility protocol.
- › Scrub the injection site in accordance with facility protocol for appropriate scrubbing and dry times.



Administer or Aspirate

- › Attach IV tubing, syringe or blood tube holder to MicroClave Clear by inserting the luer and twisting 1/4 turn, or until a friction fit is achieved.
- › Do not over-tighten a luer beyond the friction fit as this may damage both the luer and the MicroClave Clear.
- › To disconnect, grasp MicroClave Clear and then twist mating luer away from MicroClave Clear until loose. Do not hold catheter hub during disconnect as this may cause accidental removal of MicroClave Clear from hub.



Flush After Each Use

- › Flush the MicroClave Clear with normal saline or in accordance with facility protocol. After blood use, the MicroClave Clear can be flushed clean and does not require change-out.
- › Use routine flushing in accordance with facility protocol in order to maintain catheter patency.
- › Change MicroClave Clear in accordance with facility protocol and CDC Guidelines.

Functional Attributes

- › Lipid and Blood Compatible
- › Radiographic Imaging Compatible (CT Compatible)
- › Contains No Latex, Phthalate's (DEHP) or Metal Components, MRI Compatible



MicroClave Clear's saline flush option can eliminate the risk of Heparin Induced Thrombocytopenia (HIT).