


Tubifast™ WITH 2-WAY STRETCH™ TECHNOLOGY

Lightweight, tubular dressing retention

Tubifast 2-Way Stretch was the world's first lightweight elasticated tubular bandage with radial and longitudinal stretch.¹ This unique mode of action allows patients complete freedom of movement, with added comfort. Ideal for dressing retention and skin covering for any part of the body. It can also be used for patch wrapping and as an under-cast stockinette. Tubifast 2-Way Stretch holds dressings securely in place without the need for tapes, pins or ties. It is quick and easy to use, with less risk of fraying² than traditional tubular bandages.

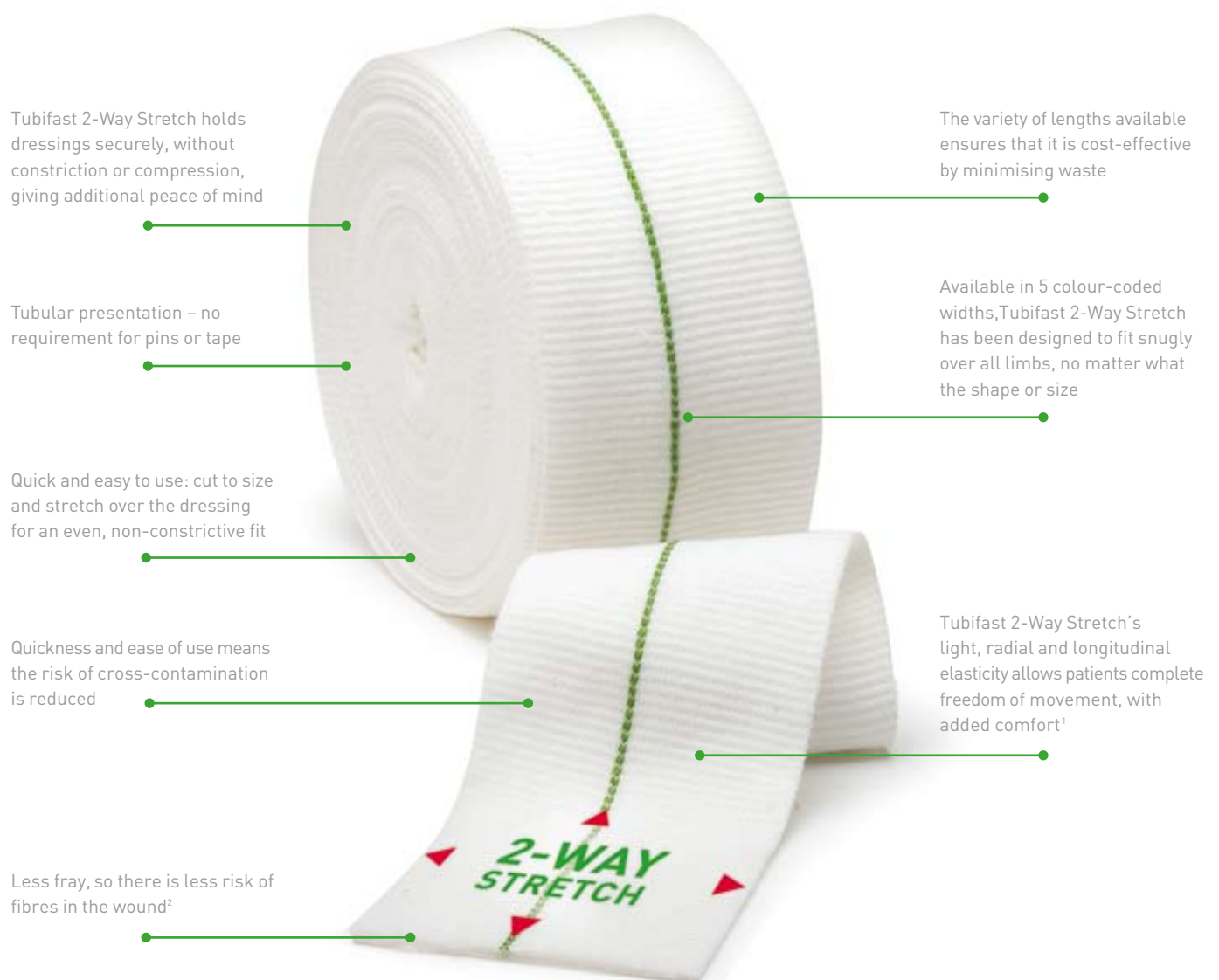

Tubifast 2-Way Stretch holds dressings securely, without constriction or compression, giving additional peace of mind

The variety of lengths available ensures that it is cost-effective by minimising waste

Tubular presentation – no requirement for pins or tape

Available in 5 colour-coded widths, Tubifast 2-Way Stretch has been designed to fit snugly over all limbs, no matter what the shape or size

Quick and easy to use: cut to size and stretch over the dressing for an even, non-constrictive fit

Tubifast 2-Way Stretch's light, radial and longitudinal elasticity allows patients complete freedom of movement, with added comfort¹

Quickness and ease of use means the risk of cross-contamination is reduced

Less fray, so there is less risk of fibres in the wound²

HOW TUBIFAST 2-WAY STRETCH WORKS

Tubifast 2-Way Stretch is manufactured from viscose with very fine elastane threads knitted into the fabric radially and longitudinally, to provide light elasticity.

Tubifast 2-Way Stretch holds dressings securely, without constriction or compression. No pins or tapes are necessary. Its two-way stretch allows patients complete freedom of movement and added comfort.

Tubifast 2-Way Stretch is available in a range of five widths, and has been designed to fit snugly over all limbs, no matter what the shape or size.


Each size is identified by a coloured line knitted into the bandage. It's quick and easy to use: cut to size and stretch over the dressing for an even, non-constrictive fit.

AREAS OF USE

Tubifast 2-Way Stretch can be used as a dressing retention and skin covering for any part of the body. It can also be used for patch wrapping and as an under-cast stockinette.

Because of its tubular construction, Tubifast 2-Way Stretch is particularly suitable for holding dressings in place on difficult areas such as stumps of amputee patients.


HOW TO USE TUBIFAST 2-WAY STRETCH

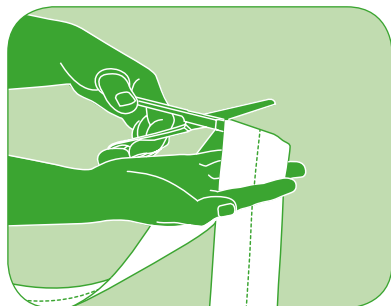

1. Choose the correct width bandage, using the quick reference colour coding, and cut to length.



2. Stretch Tubifast 2-Way Stretch over the affected area.

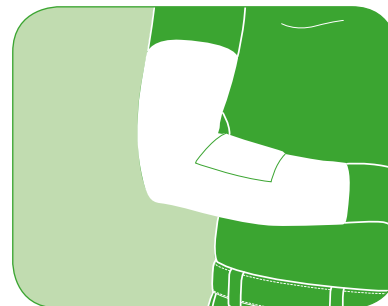

3. Position Tubifast 2-Way Stretch to cover the dressing. Illustrated, step-by-step instructions are available for more advanced applications.

BENEFITS OF TUBIFAST 2-WAY STRETCH

- Tubifast 2-Way Stretch holds dressings securely, without constriction or compression
- Its radial and longitudinal stretch allows patients complete freedom of movement, with added comfort¹
- A variety of lengths are available, ensuring cost- effectiveness by minimising waste
- Quick and easy to use: simply cut to size and stretch over the dressing for an even, non-constrictive fit
- Available in a range of quick reference, colour-coded sizes, to fit everything from small limbs to adult trunks
- It has less fray so there is less risk of fibres in the wound²

FIBRE CONTENT

Viscose, Elastane and Polyamide

WASHING INSTRUCTIONS



Natural rubber latex is not a constituent of Tubifast 2-Way Stretch or its packaging

¹ Published data shows that Tubifast 2-Way Stretch exceeds the minimum requirement of longitudinal stretch for the actions of sitting, bending and flexing. Physical testing of textiles, B. P. Saville, Woodhead Publishing, 1999.

² Data on file – Fibre Survey carried out on behalf of Mölnlycke Health Care Limited, May 2009.

Tubifast 2-Way Stretch Ordering Information

Product	Width x Length	Limb	Art. No.	PIP Code	NHS Code	
RED LINE	3.5cm x 1m	9-18 cm	2480	005-0088	-	
	Small limbs	3.5cm x 10m	9-18 cm	2434	026-8698	EGP115
GREEN LINE	5cm x 1m	14-24 cm	2481	005-0039	-	
	Small and medium limbs	5cm x 3m	14-24 cm	2485	255-1885	-
	5cm x 5m	14-24 cm	3555	255-5803	-	
	5cm x 10m	14-24 cm	2436	026-8680	EGP117	
BLUE LINE	7.5cm x 1m	24-40 cm	2482	004-9924	EGP016	
	Large limbs	7.5cm x 3m	24-40 cm	2486	255-1877	-
	7.5cm x 5m	24-40 cm	3556	255-5811	ECJ008	
	7.5cm x 10m	24-40 cm	2438	026-6924	EGP119	
YELLOW LINE	10.75cm x 1m	35-64 cm	2483	023-6539	-	
	Extra-large limbs,	10.75cm x 3m	2487	255-1869	-	
	heads, children's	10.75cm x 5m	3557	255-5829	ECJ014	
	trunks	10.75cm x 10m	2440	000-6221	EGP121	
PURPLE LINE	20cm x 1m	64-130 cm	2479	324-1452	-	
	Adult trunks	20cm x 5m	3558	324-1486	-	
	20cm x 10m	64-130 cm	2444	-	EGP017	

Tubifast 2-Way Stretch is packaged in single packs



0800 7311 876 • info.uk@molnlycke.com • www.molnlycke.co.uk

www.skincare-world.com • www.tubifast2waystretch.com

Mölnlycke Health Care Ltd, Arenson Centre, Arenson Way, Dunstable, Bedfordshire, LU5 5UL, UK

The Mölnlycke Health Care name, logo and Tubifast™ and 2-Way Stretch™ are registered trademarks of Mölnlycke Health Care AB.

©2010 Mölnlycke Health Care. All rights reserved.

UK231500803

