

Tubigrip™

MULTI-PURPOSE TUBULAR BANDAGE

- Tubigrip provides tissue support in the treatment of strains and sprains, soft tissue injuries, general oedema and ribcage injuries.



ABOUT TUBIGRIP

Tubigrip provides lasting, effective support with complete freedom of movement for the patient. Once the bandage has been applied, covered elastic threads within the fabric move to adjust to the contours of the body and distribute pressure evenly over the surface. Fitting is quick and easy.

Tubigrip stays securely in position without pins or tapes, unlike conventional crepe bandages. As it can easily be reapplied after washing, without loss of effect, Tubigrip is particularly useful for outpatients and in the community.

Tubigrip reduces wastage, as only the exact amount required is cut from the roll. Apply with a 2–3cm overlap.

Tubigrip is available in two shades – natural and beige.

THE BENEFITS OF USING TUBIGRIP

- Provides comfortable, effective tissue support
- Easy to apply and reapply
- Full range of sizes to suit most applications
- No pins or tapes needed
- Washable and reusable
- Choice of two colours for patient acceptability

HOW TO APPLY TUBIGRIP



1. Cut Tubigrip to twice length required for limb, allowing an extra 2–3cm for overlap.



2. Pull Tubigrip onto limb like a stocking.



3. Double Tubigrip back over limb. Ensure upper edge is taken 2–3cm higher up the limb than the first.

		A	B	C	D	E	F	G	J	K	L	M	
Natural Shade	0.5m	Art. No	-	1510	1513	1511	1514	1512	1515	-	-	-	-
		P.P.Case*	-	84	84	84	84	84	84	-	-	-	-
		PIP Code	-	029-3407	023-8154	029-3415	023-8246	029-3423	023-8253	-	-	-	-
		NHS Code	-	-	-	-	-	-	-	-	-	-	-
	1m	Art. No	-	1520	1521	1522	1528	1523	1524	-	-	-	-
		P.P.Case*	-	60	60	60	60	60	60	-	-	-	-
		PIP Code	-	029-3456	029-3464	029-3472	048-9971	029-3480	029-3498	-	-	-	-
		NHS Code	-	-	-	-	-	-	-	-	-	-	-
	10m	Art. No	1435	1436	1443	1437	1434	1438	1439	1440	1441	1442	1444
		P.P.Case*	44	32	28	28	24	20	12	8	6	4	4
		PIP Code	029-3258	029-3266	029-3282	029-3290	023-1969	029-3316	029-3324	029-3332	029-3340	029-3357	026-6569
		NHS Code	EGA011	EGA013	EGA015	EGA017	EGA019	EGA021	EGA023	EGA025	EGA027	EGA029	-
Beige Shade	1m	Art. No	-	-	1545	1546	1547	1548	-	-	-	-	
		P.P.Case*	-	-	60	60	60	60	-	-	-	-	
		PIP Code	-	-	006-8833	006-8973	006-9146	006-9153	-	-	-	-	
		NHS Code	-	-	-	-	-	-	-	-	-	-	
	10m	Art. No	-	1449	1450	1451	1448	1452	1453	-	-	-	-
		P.P.Case*	-	32	28	28	24	20	12	-	-	-	-
		PIP Code	-	029-3365	029-3373	029-3381	023-1977	029-3399	023-8386	-	-	-	-
		NHS Code	-	-	-	-	-	-	-	-	-	-	-

*Abbreviation of Pieces Per Case

KEY

A - Infant feet and arms

B - Small hands and arms

C - Medium arms, small ankles

D - Large arms, medium ankles, small knees

E - Large ankles, medium knees, small thighs

F - large knees, medium thighs

G - Large thighs

J - Small trunks

K - Medium trunks

L - Large trunks

M - X-Large trunks

