



Prosys™ Urine Meter

The Prosys™ Urine Meter has been thoughtfully designed and manufactured to support high quality treatment for patients and ease of use for healthcare professionals.

The Prosys™ Urine Meter is available in the hospital sector for consistent and accurate measuring of urine output.



- Low profile design to avoid accidental contact with the floor
- Sterile and non-latex
- Needle-free sample port eliminates the risk of sharps injuries
- T-tap kennel helps protect the tap from external bacteria
- Long kink-resistant tubing
- 50ml burette
- 400ml meter with 5ml increments
- 2600ml urine collection bag
- Flexible hanging options for diverse bed designs

Description	Size	CliniSupplies Product Code	Supply Chain Code
Prosys™ Urine Meter	400ml capacity with 2600ml drainage bag	P400	FWW050



BEST PRACTICE

Do:

- Keep meter hanging upright at all times
- Always note measurements before moving patient or meter
- Keep the drainage bag lower than the catheter and the patient's bladder
- Keep meter away from visitors to the bedside – accidental tilting of the bag can lead to loss of measurements
- Hold spine and lift meter to facilitate emptying

Don't:

- Lie meter flat on patient's bed
- Forget to take measurements until the patient or meter have been moved
- Allow the tubing to loop or kink

WHO 5 Moments for Hand Hygiene

