
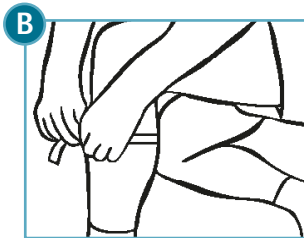
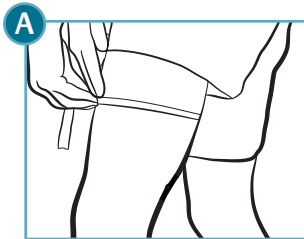


Prosys®

Leg Bag Sleeve

Prosys Leg Bag Sleeve has a slit pocket at the top, where the urine drainage bag can be easily inserted and remain in place even at full capacity.

At the bottom of bag sleeve an outlet has been designed to allow the positioning of the outlet tap. This allows the easy emptying of your collection system without removing your drainage bag from the bag sleeve.

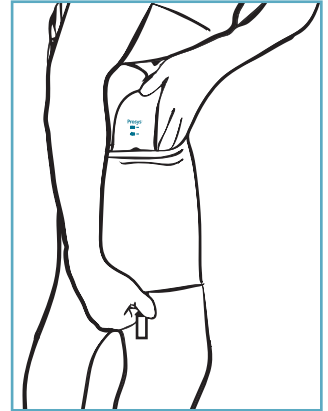

To choose the correct **Prosys Leg Bag Sleeve** size, please measure the circumference of the:

- A** mid-point of your thigh, if you wish to place your urine drainage bag on the upper part of your leg or
- B** widest-point of your calf, if you wish to place your urine drainage bag on the lower part of your leg

WASHING INSTRUCTIONS



The **Prosys Leg Bag Sleeve** is made of Polyamide 92%, Elastane 7%, Viscose 1% and is machine washable.

LATEX FREE



CLINISUPPLIES
Ideas into solutions

**For more information please contact
CliniSupplies on 0208 863 4168**

CliniSupplies Ltd., Qualitas House, 100 Elmgrove Road, Harrow, Middlesex HA1 2RW UK.
Telephone: 0208 863 4168 Fax: 0208 426 0768 Email: info@clinisupplies.co.uk
www.clinisupplies.co.uk

Prosys is a trademark of CliniSupplies Ltd. CS299_1113