


## INSTRUCTIONS FOR USE



1. Remove the iPro pad from the backing paper, using the coloured tab

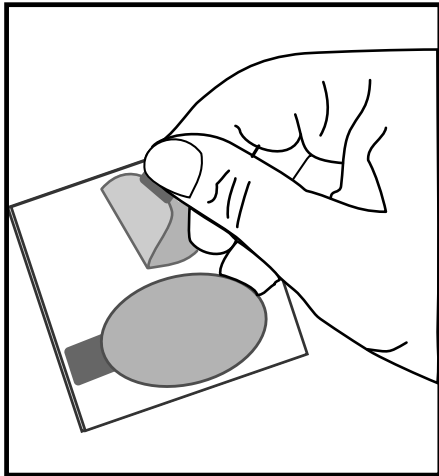

2. Holding the coloured tab, position the iPro over the eye.  
- Ensure the eyelid is closed and fully covered.



3. Press gently to ensure the eye protector is securely in place - Repeat procedure for the other eye.

#### CONTRAINDICATIONS:

There are no known contraindications for this device.

#### WARNINGS & PRECAUTIONS:

The device is only to be used by suitably qualified clinicians, who are experienced with the specific techniques required for these medical devices.

**This device is single-use. Do not re-use.**

**Caution:** Use tabs only for placement and removal of the device. Take care upon removal of the device from the patient's eye area.




**Fannin (UK) Limited**  
**Westminster Industrial Estate**  
**Repton Road, Measham, Derbyshire,**  
**DE12 7DT, United Kingdom**

**T:** +44 (0)1453 874 130

**E:** salesll@fannin.eu

**W:** www.fannin.eu

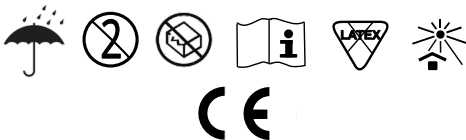

RM01231

Revision:02



**Device Name:** iPRO & iPRO ELITE - SKINTACT Eye Protector

#### INDICATIONS FOR USE:

A mild adhesive eye protector, to be used as an alternative to adhesive tape during surgery.

#### DEVICE DESCRIPTION:

Designed to reduce the risk of corneal abrasions & other common hazards encountered during anaesthesia.

The eye protector helps provide a consistent and even pressure on the eyelid. By keeping the eyelid shut, this product helps maintain the natural tear film on the cornea during the procedure.

