

# Pre-Op Glove


Ref: STMT200

# Sticky Mitt



## To facilitate the removal of hair and lint from the patients skin

- Provides an efficient way of removing loose hairs and debris during pre-op surgery preparation
- Quick application with a large adhesive area.
- Optimised adhesion reduces the risk of skin irritation.
- Can be used on both curved and flat areas
- Can help to surgical site contamination risk.
- Helps reduce the risk of post procedural contamination/infection
- Latex free.
- Easy disposal after use

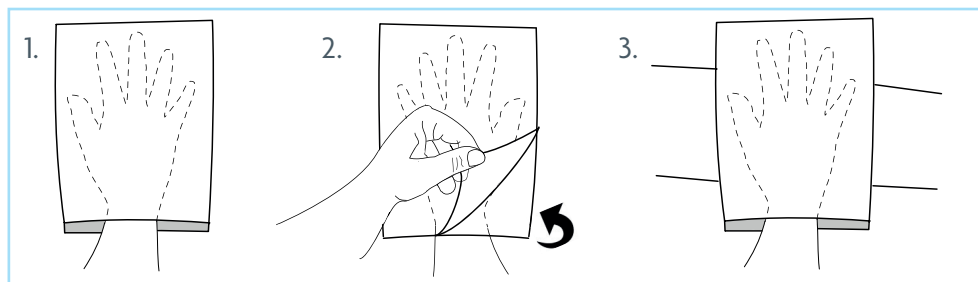

## Instructions for use:

1. Insert hand into the open end, with the palm facing the backing paper.
2. Turn over and gently peel the backing paper
3. Remove the loose hairs or lint with the adhesive surface.

## For more information, please contact us at:

**Fannin UK Ltd**  
Fernhouse - Suite 3  
Daniels Industrial Estate  
Bath Road, Gloucestershire  
GL5 3TJ, United Kingdom  
T: +44 (0) 1453 874 130  
E: salesll@fannin.eu

**Fannin Ltd.**  
Fannin House  
South County Business Park  
Leopards Town  
Dublin 18  
T: +353 (0)1 290 7000  
E: info@fannin.eu