

How Airtraq simplifies Video Intubation

- Airtraq's channel simplifies ETT insertion reducing intubation time
- No stylet needed. Avoids potential injuries.
- Eliminates stylet costs
- 90° shape minimizes hyperextension and reduces force required
- Reduces hemodynamic reactivity
- Facilitates intubation from any position (face to face etc)
- High intubation success rate in difficult airways
- Easier to learn than other video laryngoscopes

Visualisation Options for any Airtraq model



Product code	Description	ETT size	Mouth opening	Units per box	NHSSC code
SP					
A-011	Airtraq SP Regular (size 3)	7.0 - 8.5	16mm	6	FDS2530
A-021	Airtraq SP Small (size 2)	6.0 - 7.5	15mm	6	FDD2531
A-031	Airtraq SP Paediatric (size 1)	4.0 - 5.5	11.5mm	6	FDD2532
A-041	Airtraq SP Infant (size 0)	2.5 - 3.5	11mm	6	FDD2533
A-061	Airtraq SP Adult Nasal	-	15mm	6	FDD2535
A-071	Airtraq SP Double Lumen	35 - 41 Fr.	19mm	6	FDD2536
AVANT					
A-501	Airtraq Avant Optics	-	-	1	FSM6157
A-511	Airtraq Avant Regular Blade (size 3)	7.0 - 8.5	17mm	50	FSM6158
A-521	Airtraq Avant Small Blade (size 2)	6.0 - 7.5	17mm	50	FSM6159
A-590	Airtraq Avant Docking Station	-	-	1	FSM6160
ACCESSORIES					
A-307	Airtraq Phone Adapter	-	-	1	FSM6155
A-309	Airtraq Phone Protection Sleeve	-	-	100	FSM3654
A-390	Airtraq Wi-Fi Camera	-	-	1	FSM3778
A-390-KIT	Airtraq Wi-Fi Camera Kit	-	-	1	FSM3779

Where and when you need it



www.fannin.eu

Fannin (UK) Ltd.
Westminster Industrial Estate
Repton Road, Measham,
Swadlincote, Derbyshire,
DE12 7DT
T: +44 (0) 1530 514566
E: serviceuk@fannin.eu

FANUK18-4407



Airtraq
GUIDED VIDEO INTUBATION

Simple, effective,
affordable video
laryngoscopy

Two complementary models
for one complete solution



Airtraq AVANT

Reusable Optics + Disposable Blades

Minimum cost per intubation.
VL for frequent use.



Airtraq SP

Fully Disposable

One piece. Ready to use.
VL for remote locations and emergencies.



- Keep midline
- Do not insert too deep
- Lift gently
- Twist Airtraq to centre vocal cords
- Corkscrew ETT if needed

Airtraq improves routine intubations and it is specially effective for:

- Cervical immobilization
- In combination with a fibroscope
- Obese
- Frontal or lateral intubations
- Awake
- Poor dentition

Airtraq SP
GUIDED VIDEO INTUBATION

Adults
Paediatrics
Double Lumen
Nasal (no channel)



Adult regular
7.0 to 8.5 ETT
M.O.*: 16mm

Adult small
6.0 to 7.5 ETT
M.O.*: 15mm

Double lumen
35 to 41 Fr.
M.O.*: 18mm

Adult Nasal
M.O.*: 15mm

**NEW WiFi
Camera**



iOS / Android
Mac / Windows



Paediatric
4.0 to 5.5 ETT
M.O.*: 12mm

Infant
2.5 to 3.5 ETT
M.O.*: 11mm

Adult regular
7.0 to 8.5 ETT
M.O.*: 17mm

Adult small
6.0 to 7.5 ETT
M.O.*: 17mm




Reusable optics

Disposable blades

Docking

Phone Adapter

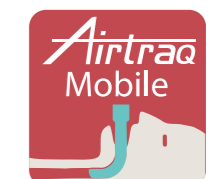

iOS / Android



Airtraq Avant