


Airtraq

GUIDED VIDEO INTUBATION

Simple, effective, affordable video laryngoscopy

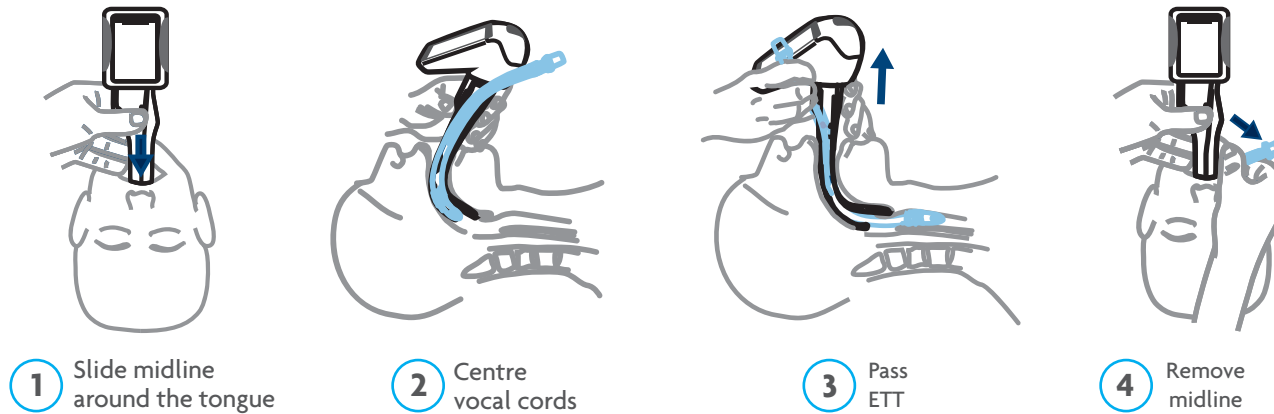

Useful tips

- Keep Airtraq handle in the midline
- Do not insert too deep
- Lift gently
- Twist Airtraq to centre vocal cords
- Corkscrew ETT if needed

Airtraq Avant



Adult regular
7.0 to 8.5 ETT
M.O.*: 17mm
A-511

Adult small
6.0 to 7.5 ETT
M.O.*: 17mm
A-521



Camera, reusable optics
& docking station

A-390, A-501 & A-590



iOS / Android
Mac / Windows



Airtraq sp

GUIDED VIDEO INTUBATION



Adult regular
7.0 to 8.5 ETT
M.O.*: 16mm
A-011

Adult small
6.0 to 7.5 ETT
M.O.*: 15mm
A-021

Paediatric
4.0 to 5.5 ETT
M.O.*: 12mm
A-031

Infant
2.5 to 3.5 ETT
M.O.*: 11mm
A-041

Adult Nasal
M.O.*: 15mm
A-061

Double lumen
35 to 41 Fr.
M.O.*: 18mm
A-071