

CBC VECOS® Tourniquet

- 100% Hygiene & 100% Comfort
- Single use




Hygienic. Gentle. Easy. Safe.



CBC VECOS® Tourniquet

The only single use tourniquet
with defined compression force

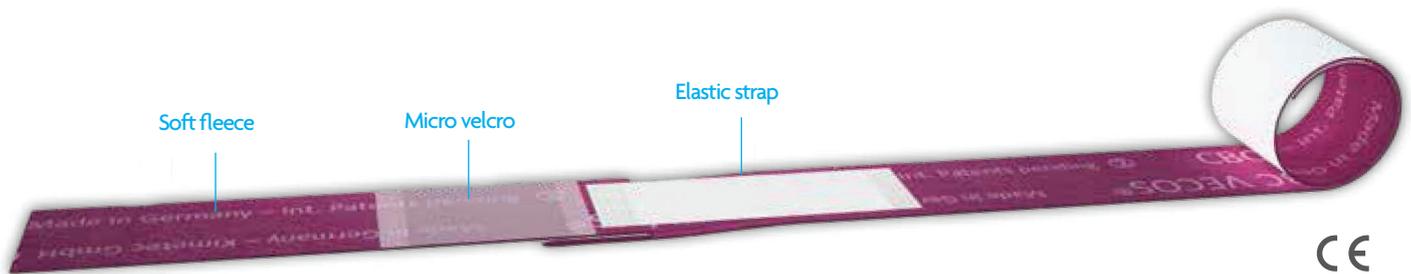

100% Hygiene & 100% Comfort

No laborious cleaning or disinfection

- Single use product - no cleaning or disinfection necessary
- No cross-contamination with MRSA and other germs

Comfort for medical professionals

- Easy handling, even when wearing gloves
- Usable in dry, wet or cold environments
- Micro velcro for easy closing and opening
- Particularly low weight (2.5g/piece)

Suitable for every patient's arm (from Ø 4 cm)

- Small: 3cm wide, 50cm long
- Universal: 3.5cm wide, 65cm long

Comfort for every patient & skin type

- Velvety soft fleece – no bruising, lacing or constricting of the skin
- Free from latex, silicone and plasticisers

Cost-effective

- Also usable as compress after venipuncture
- No reprocessing costs

Made in Germany – International patents pending

VEIN COMPRESSION SYSTEM



Testimonial by Prof. Dr. Popp from HyKoMed GmbH

“Our experience has shown that especially ‘simple’ medical devices like tourniquets can cause problems regarding the implementation of necessary and required hygiene. For this purpose, your disposable tourniquet is the optimal solution.”

100% Stretching = 100% Compression


Optimal compression force (approx. 20 mmHg)

- By extension of the elastic strap, the optimal compression force is reliably achieved
- Defined compression force – supporting the user
- Complications of venipuncture (e.g. rupture) can be reduced

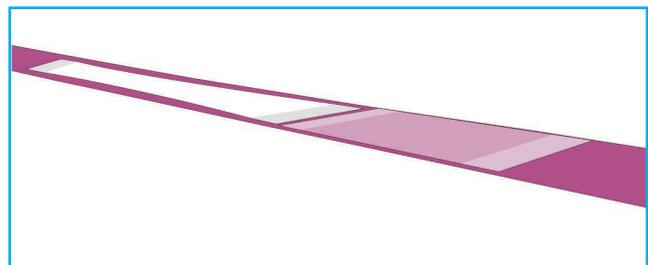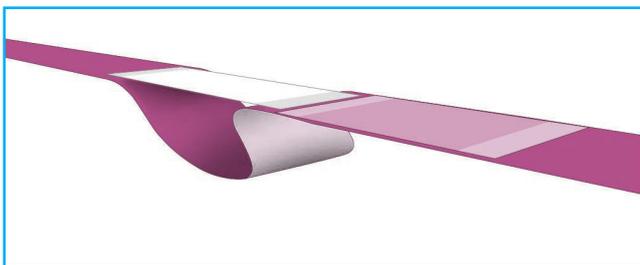


CBC VECOS® Tourniquet


The only single use tourniquet with defined compression force

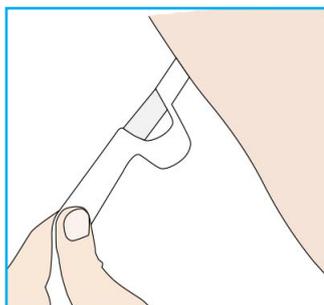


1. Place unprinted side on the patient's arm.

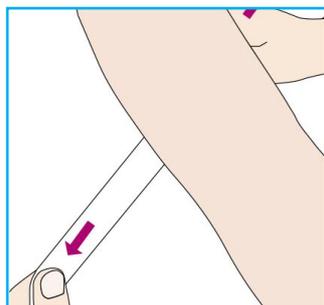

2. Build up compression force by extending the tourniquet completely.

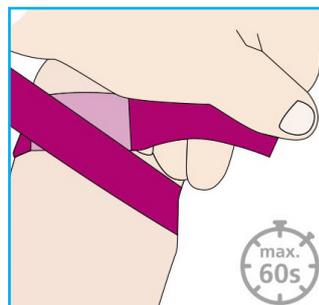

3. Maintain the tension!
Place tightly around the arm and attach in the velcroed area.

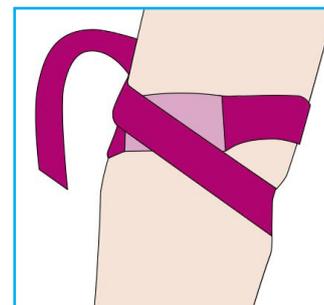

4. Simply pull to release the strap.

Ordering information

Product code	Size	Product description	Units per box
521001 V 125	Small	Strap 3cm wide, 50cm long	125 pieces in dispenser box
521001 B 125	Small	Strap 3cm wide, 50cm long	125 pieces in poly bag
521002 V 100	Universal	Strap 3.5cm wide, 65cm long	100 pieces in dispenser box
521002 B 100	Universal	Strap 3.5cm wide, 65cm long	100 pieces in poly bag

Do not reuse to avoid cross-contamination and dispose of in accordance with regulations.

Training video on correct application:



SILICONE-FREE

DEHP-FREE

