

Surgical Clippers

New generation surgical clipper with advanced Li-ion battery technology

- Ergonomic design with unique twistable shaving head
- Disposable Universal blade and Neuro blade available
- Superior cutting performance for all types of hair
- Non-slip rubber handle for safe operation
- Rechargeable lithium battery with an operating time of 150 minutes and overcharge protection
- Waterproof according to IPX-7 standard
- Suitable for cleaning with a disinfectant
- Comes with a 2 year warranty

Digital LED battery indicator

Can see the exact battery charge level at all times and avoid losing charge during use.

Twistable shaving head

Shaving head can be turned at a 45° angle to the left and right, allowing for safe operation in difficult areas e.g. genital area

Easy insertion & ejection of blade

Single blade can be easily inserted with two fingers on the shaving head, and easily ejected with the push of a thumb.




Battery charger

Easily mounted on the wall.




Protection against cuts

Design prevents blade from coming into contact with the skin, avoiding cuts and lesions.

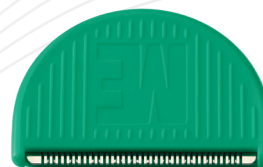

Neuro blade

Designed for coarse or dense hair in areas such as the scalp or chest.

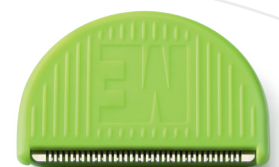

Universal blade

Designed for most body area hair and cutting needs.

Recommended procedure

1. Insert disposable shaving head

Remove the disposable shaving head from the pack. Install the new blade on the hair cutting head and slide the blade back until it snaps into place and all edges are flush.

2. Cutting hair

Hold the handle of the clipper like a pen, with the ME logo on top. Stretch the skin, put the shaving head flat on the skin and run the clipper against the direction of hair growth.

3. Remove the blade

Turn off the surgical clipper. Hold it with the blade pointing down over a waste container and push the blade with your thumb off the clipper.

4. Clean the clipper hand set

Wash the handle with water, then dry it. Clean the surgical clipper with a surface disinfectant or disinfectant towel.

5. Loading & storing

Set the surgical clipper upright in the charger, so that it will be ready for use when next required.

Ordering Details

Product Code	Product Description	Units per Box	NHSSC Code
SC004	Clipper unit	2	FCC85656
SC010	Clipper charger (UK plug)	1	FCH85002
SC003	Neuro clipper blades (for dense hair)	50	FCH85003
SC002	Universal clipper blades (for most body hair)	100	FCH85000
IH-SC004	Starter Pack (including unit, charger, neuro blades and universal blades)	1 Case	FCH85001

